

Fast. Easy. Accurate.

DELUXE BLOOD PRESSURE MONITOR UA-651



**WORLD CLASS™
MONITOR**



Clinically
Validated
for Accuracy

UA-651, part of the World Class Monitor family.

The UA-651 Deluxe Blood Pressure Monitor is easy to use and comes standard with just the right number of features. 15 memory recall provides ample storage for home use or while traveling. Systolic, diastolic and pulse rate are all shown simultaneously for easy review. The UA-651 has A&D Medical's proprietary Pressure Rating Indicator™, providing a snapshot of the user's blood pressure classification according to the US JNC7 guidelines. Irregular Heartbeat Detection also comes standard. This key feature alerts the user to the presence of an irregular heartbeat while still providing an accurate reading.

All the benefits of speed and accuracy are at the touch of a single button. The UA-651 Deluxe Blood Pressure Monitor is also one of our most efficient devices. When using alkaline batteries a user can expect approximately 700 readings before having to change batteries. A battery life indicator appears on the screen preventing any surprises.

The standard cuff of this product is A&D Medical's proprietary AccuFit™ Plus cuff which is designed to fit the width of most people's arm with 8.6" - 16.5" (22 - 42cm) circumference range. The UA-651 also works with an optional medium cuff and a large cuff. We've got you covered with the strongest cuff warranty in the industry, three years. The monitor comes backed with a five-year warranty.


- **Pressure Rating Indicator** – Providing a snapshot of the user's blood pressure classification according to the US JNC7 guidelines.
- **Irregular Heartbeat Detection** – Alerts the user to the presence of an irregular heartbeat while still providing an accurate reading.



- 30 Memory Recall
- Shows systolic diastolic and pulse rate
- Pressure Rating Indicator™
- Irregular Heartbeat Detection
- Battery life indicator
- Single button operation

UA-651 Deluxe Blood Pressure Monitor

To obtain an accurate reading, it is important to use the correct cuff size. A cuff that is too large will produce a reading that is lower than the correct blood pressure; a cuff that is too small will produce a reading that is higher than the correct blood pressure. The UA-651 Deluxe Blood Pressure Monitor works with many A&D Medical and LifeSource brand cuffs.

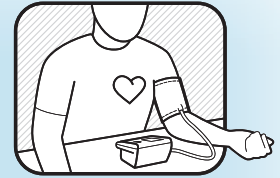

PART NUMBER	UPC NUMBER	ACCESSORY
UA-651	0-93764-60334-1	AccuFit™ Plus cuff 8.6" - 16.5" (22 - 42cm)

Each unit comes with a bilingual instruction manual and quick start card.

Accessories

PART NUMBER	UPC NUMBER	ACCESSORY
UA-290	0-93764-60298-6	SlimFit Medium Cuff 9.0"-14.6" (23-37cm)
UA-291	0-93764-60299-3	SlimFit Large Cuff 12.2"-17.7" (31-45cm)
UA-420	0-93764-60343-3	AccuFit™ Plus cuff 8.6" - 16.5" (22 - 42cm)

Technical Specifications

Dimensions	3.8" (w) x 5.1" (d) x 2.7" (h) (97mm x 130mm x 69mm)
Weight	8.1 oz. (230g) without batteries
Display	34.9 x 54.2mm (view area)
Memory	Last 30 measurements
Pressurization	Automatic, using micropump
Deflation	Constant air release valve system
Type	Oscillometric
Accuracy	Pressure: ±3 mmHg. Pulse: ±5%
Pressure Range	Systolic pressure: 60 - 279 mmHg - Diastolic pressure: 40 - 200 mmHg Pulse: 40 - 180 beats / minute
Power Source	4 x 1.5v batteries (R6P, LR6 or AA)
Recommended Operating Environment	50°F to 104°F (+10°C to +40°C). 800 to 1060hPa
Recommended Storage Environment	-4°F to 140°F (-20°C to +60°C).

Blood pressure measurements determined with this device are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultation method within the limits prescribed by the American National Standards Institute (ANSI/AAMI SP10) for electronic sphygmomanometers. Manufactured by an ISO 9000 and ISO 13485 certified company. Engineered in Japan. Made in China.

Packaging and Shipping Specifications

Selling Unit (Box)	Dimensions (L x D x H)	Weight	Individual Item Cube
UA-651	5.2 x 4.1 x 6.7 in	20.1 oz	0.08
Unit Case (10 units)	W x D x H (inches)	WT (lb.)	Case Cube (cubic feet)
UA-651	22 x 10.5 x 17	14.0	1.13

Ask your A&D Medical Sales Representative about other A&D Medical & Wellness Connected products.

- Blood Pressure Monitors
- Digital Thermometers
- Personal Scales
- Activity Monitors



LifeSource, Pressure Rating Indicator, SlimFit, SmoothFit, and World Class Monitor are trademarks and registered trademarks of A&D Company and/or A&D Medical. ©2013 A&D Medical. All rights reserved.

LIT-UA-651 01/2015

For questions about A&D Medical products, call us toll-free at 1-888-726-9966 or visit one of our websites at:
www.andmedical.com
www.WellnessConnected.com
www.LifeSourceOnline.com