# ACCUFIT™ PLUS BLOOD PRESSURE CUFF


1. This cuff is intended for use with A&D Medical or LifeSource® blood pressure monitors.
2. Intended for upper arm use.
3. Intended for use at home by those who are eighteen (18) years or older, to monitor blood pressure (systolic and diastolic) and pulse rate.
4. Not made with natural rubber latex.
5. Avoid tightly folding of this cuff or storing the hose tightly twisted for long periods, as such treatment may shorten the life of the components.
6. This cuff is not water resistant. Prevent rain, sweat, water and moisture of any type from soiling this cuff.

## BLOOD PRESSURE TAKING TIPS

- Relax for several moments before measurement.
- Do not smoke or ingest caffeine at least 30 minutes prior to measurement.
- Remove constricting clothing and place cuff on bare arm.
- Unless your physician recommends otherwise, use left arm to measure pressure.
- Do not talk during measurement.
- Do not cross legs and keep your feet flat on the floor.

## Now you are ready. Follow these simple steps:

1. Sit comfortably with your left arm resting on a flat surface so that the center of your upper arm is at the same height as your heart.

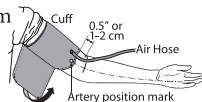
2. Lay left arm on the table, palm up and wrap the cuff around the upper arm, about 0.5" (1-2 cm) above the inside of the elbow, as shown, place the cuff directly against the skin, as clothing may cause a faint pulse, and result in a measurement error.

3. Secure cuff around arm. The cuff should be snug but not too tight.

4. Make sure cuff is aligned At heart level

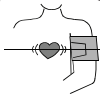# AND

A&D Medical

1-888-726-9966

[www.andmedical.com](http://www.andmedical.com)

© 2014 A&D Medical. All rights reserved. Specifications subject to change without notice. UA-420-QSC-ENG-092014.

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