UA-705 QUICK START CARD


1. Slide cuff onto arm so that the bottom edge of cuff is just above the elbow crease. Then position tube off-center toward the inner side of arm in line with the little finger


2. Fasten the cuff tight enough so you can slide two fingers under it.
3. Sit comfortably with arm resting on a flat surface, palm up.
4. Press START button. (Battery installation required. Please see manual for instructions.)

5. Your average blood pressure reading will appear followed by a zero, indicating that monitor is ready for measurement.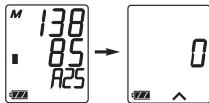

6. Press the exhaust valve until the ▲ mark is displayed. Squeeze bulb repeatedly until the ▲ mark disappears.

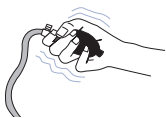
7. Sit quietly during measurement.

8. Your systolic and diastolic pressure and pulse rate will appear in the digital display.

9. Press exhaust valve on the bulb to release remaining air from cuff after measurement is complete.

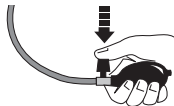BLOOD PRESSURE TAKING TIPS

- Measure at the same time every day.
- Sit in the same chair/position.
- Do not cross legs and keep your feet flat on the floor.
- Relax for 5 minutes before measurement.
- Use the correct cuff size to get an accurate reading.
- Sit still during measurement – no talking, eating or sudden movements
- Record your measurement in a logbook.

1-888-726-9966

www.LifeSourceOnline.com

LIFE SOURCE[®]

LifeSource is a trademark of A&D Medical.

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©2008 A&D Medical. All rights reserved. 8/08.