



ENGLISH

**1. Initialize and install the batteries.**

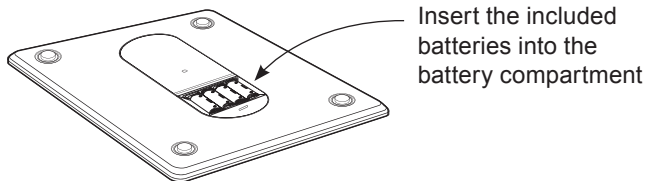

**2. Download the free "A&D Connect" app.**

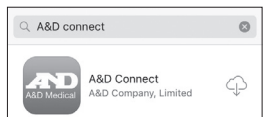

**3. Account registration, device setup and pair-up.**

Follow the A&D Connect app instructions for account registration and device setup/connection process.

**Account registration**

*(Create an account; App will save your data on your account and calculate your BMI)*



**Guest mode**

*(Reading is saved on device memory but not in your account)*

**3a. Device setup**


Select UC-350BLE from device setup screen.




**UC-350BLE**  
Weight Scale

**3b. Pair-up**

Ensure the *Bluetooth*® function of the mobile device is ON. Press and hold UNIT button in the back of the scale to start pair-up.

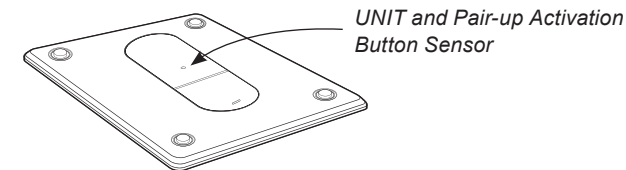

Symbol  $\square$   $\square$  and symbol  $\square$   $\square$  will be shown on the LCD alternatively, indicating pair-up is proceeding.

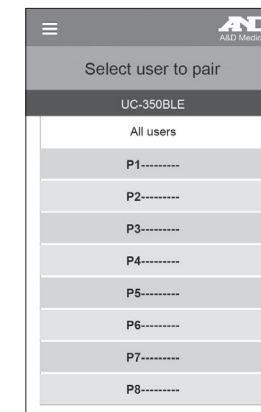

Select User ID among P1 to P7.

**NOTE: Remember your "P#" to ensure data is stored under correct user.**

Symbol [ ] will appear on the LCD, if pairing is successful.  
*See the manual for more details.*

#### 4. Start measuring

**Be sure to select your “P#” on the scale. Guest User should select “P8”.**

- On the scale, press and hold “SET” button for about 3s to enter setting mode. Once “P1” flashes, press the function button ▲ or ▼ to select your “P#” among P1 to P8. Press “SET” button to lock it.
- Step on the platform when the scale displays “0.0”. Stand still until the unit (lb/kg) stop flashing.
- A beep sound will indicate to step off and the scale will display your weight value
- Data transmission will process automatically if paired successfully.

*Refer to Data Transmission in the manual for more details.*



#### 5. Multiuser

- Up to 8 people can use and store reading on the scale.
- **Be sure to select your “P#” on the scale. Guest User should select “P8”.**
- Follow the steps above to start measuring

#### 6. Offline measurement

Even when you don't have your phone, the weight data is stored in your ID memory and transmitted next time the scale is connected with your phone (max 30 memory).

**Be sure to select your “P#” on the scale. Guest User should select “P8”.**

- Position the scale on a flat, hard surface.
- Step onto the scale platform and remain still until the unit (lb/kg) stop flashing.
- A beep sound will indicate to step off and the scale will display your weight value
- The scale will automatically turn off after a few seconds.



**AND**  
A&D Medical

1-888-726-9966 • [www.andmedical.com](http://www.andmedical.com)

©A&D Medical. All rights reserved. iPad, iPhone and iPod are trademarks of Apple, Inc. Android and Google play are trademark of Google, Inc. Specifications subject to change without notice.