



UT-302 QUICK START CARD


1. Press and hold the SCAN button for one second.

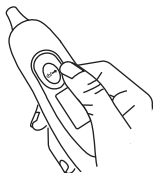



Wait for the last measurement taken and the Ready symbol to appear (thermometer will beep when ready).

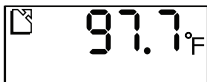

2. Press and hold the SCAN button. Insert thermometer tip gently into the ear.

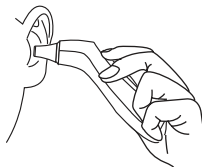

3. Release the SCAN button and wait for beep. Remove the thermometer from ear canal. The temperature is displayed.

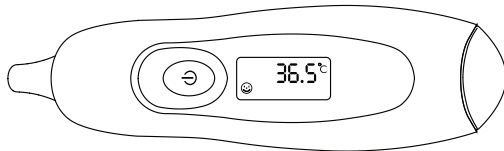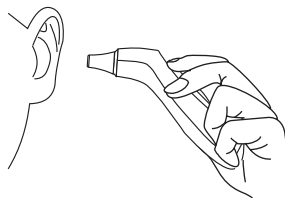


AND
A&D Medical


1-888-726-9966 (USA)
www.andmedical.com

UT-302 TARJETA DE INICIO RAPIDO


1. Presione y sostenga el botón de SCAN por un segundo.

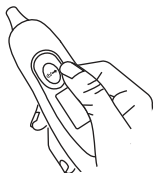



Espere que aparezca en la pantalla su última medición y el símbolo de "listo" (el termómetro sonará cuando esté listo).

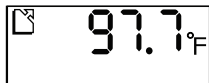

2. Presione y sostenga el botón de SCAN. Inserte la punta del termómetro suavemente dentro del oído.

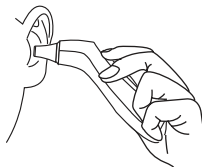

3. Suelte el botón de SCAN y espere para de sonido. Retire el termómetro de canal de oído. La temperatura será mostrada en la pantalla.

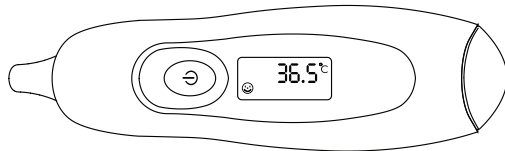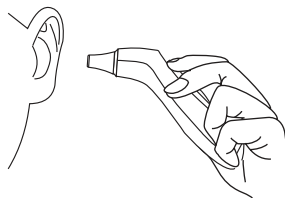

AND
A&D Medical

1-888-726-9966 (USA)
www.andmedical.com