ESSENTIAL

BLOOD PRESSURE MONITOR

WRIST

UB-525

Instruction Manual

Dear Customers

Congratulations on purchasing a state-of-the-art A&D blood pressure monitor, one of the most advanced monitors available today. Designed for ease of use and accuracy, this device will facilitate your daily blood pressure regimen. We recommend that you read through this manual carefully before using the device for the first time.

Preliminary Remarks

- The device is designed for use on adults, not newborns or infants.
- Environment for use. The device is for use to operate by yourself in the home healthcare environment.
- This device is designed to measure blood pressure and pulse rate of people for diagnosis.

Precautions

- Precision components are used in the construction of this device. Extremes in temperature, humidity, direct sunlight, shock or dust should be avoided.
- Clean the device with a dry, soft cloth or a cloth dampened with water and a neutral detergent. Never use alcohol, benzene, thinner or other harsh chemicals to clean the device.
- Avoid tightly folding the cuff for long periods, as such treatment may shorten the life of the components.
- The device is not water resistant. Prevent rain, sweat and water from soiling the device.
- Measurements may be distorted if the device is used close to televisions, microwave ovens, cellular telephones, X-ray or other devices with strong electrical fields.
- Used equipment, parts and batteries are not treated as ordinary household waste, and must be disposed of according to the applicable local regulations.

Precautions

- When reusing the device, confirm that the device is clean.
- Do not modify the device. It may cause accidents or damage to the device.
- To measure blood pressure, the wrist must be squeezed by the cuff hard enough to temporarily stop blood flow through the artery. This may cause pain, numbness or a temporary red mark to the wrist. This condition will appear especially when measurement is repeated successively. Any pain, numbness, or red marks will disappear with time.
- Wireless communication device, such as home networking devices, mobile phones, cordless phones and their base stations, walkie-talkies can affect this blood pressure monitor. Therefore, a minimum distance of 0.3 meters should be kept from such devices.
- Measuring blood pressure too frequently may cause harm due to blood flow interference. Check that the operation of the device does not result in prolonged impairment of blood circulation, when using the device repeatedly.
- Clinical testing has not been conducted on newborn infants and pregnant woman. Do not use on newborn infants or pregnant woman.
- If you have had a mastectomy, please consult a doctor before using the device.
- Do not let children use the device by themselves and do not use the device in a place within the reach of infants. It may cause accidents or damage.
- There are small parts that may cause a choking hazard if swallowed by mistake by infants.
- Do not touch the batteries and the patient at the same time. That may result in electrical shock.
- In the case of single components failure enclosure of near cuff may become hot and potentially cause malfunction.

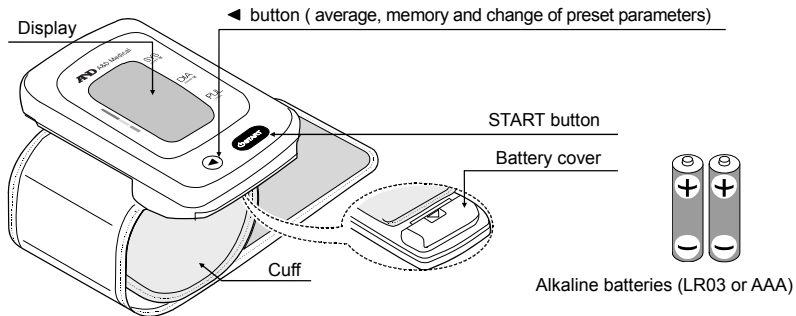
Precautions

- Use of accessories not detailed in this manual may compromise safety.
- Should the battery short-circuit, it may become hot and potentially cause burns.
- Allow the device to adapt to the surrounding environment before use (about one hour).
- Do not inflate without wrapping the cuff around your wrist.
- Do not apply the device to a wrist with other medical electrical equipment attached. The equipment may not function properly.
- People who have a severe circulatory deficit in the arm must consult a doctor before using the device to avoid medical problems.
- Do not self-diagnose the measurement results and start treatment by yourself. Always consult your doctor for evaluation of the results and treatment.
- Do not apply the device on a wrist with an unhealed wound.
- Do not apply the device on an arm receiving an intravenous drip or blood transfusion. It may cause injury or accidents.
- Do not use the device where flammable gases such as anesthetic gases are present. It may cause an explosion.
- Do not use the device in highly concentrated oxygen environments, such as a high-pressure oxygen chamber or an oxygen tent. It may cause a fire or explosion.

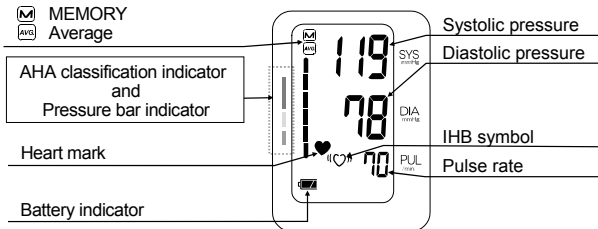
Contents

Parts Identification	E-6
Symbols	E-7
Using the Monitor	E-10
Measurements	E-13
Recalling the Memory Data	E-14
Deleting Data Stored in Memory.....	E-14
What is Irregular Heartbeat?	E-15
Pressure Bar Indicator	E-15
AHA Classification Indicator	E-16
About Blood Pressure	E-17
Troubleshooting	E-19
Maintenance	E-20
Technical Data	E-20
Warranty	E-22

Parts Identification




Display

Symbols

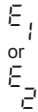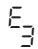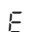Symbols that appear on the display

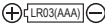
Symbols	Function/Meaning	Recommended Action
	Appears while measurement is in progress. It blinks when the pulse is detected.	Measurement is in progress. Remain as still as possible.
	Irregular Heartbeat symbol (IHB) Appears when an irregular heartbeat is detected during measurement	_____
	Previous measurements stored in memory	_____
	Average data	_____
	The battery power indicator during measurement.	_____
	The battery is low when it blinks.	Replace all batteries with new ones when the mark blinks.
	Unstable blood pressure due to movement during measurement	Take another measurement. Remain still during measurement.
	The systolic and diastolic values are within 10 mmHg of each other.	Apply the cuff correctly, and take another measurement.
	The pressure value did not increase during inflation.	Apply the cuff correctly, and take another measurement.
	The cuff is not applied correctly.	
	PUL DISPLAY ERROR The pulse is not detected correctly.	

Symbols

Symbols	Function/Meaning	Recommended Action
E E	Blood pressure monitor internal error	Remove the batteries and press the START button, and then install the batteries again. If the error still appears, contact the dealer.
E 9		

Symbols


Symbols that are printed on the device

Symbols	Function/Meaning
	Standby and Turn the device on.
SYS	Systolic blood pressure in mmHg
DIA	Diastolic blood pressure in mmHg
PUL	Pulse per minute
	Battery installation guide
	Direct current
	Type BF: Device, cuff and tubing are designed to provide special protection against electrical shocks.
SN	Serial number
	Refer to instruction manual/booklet
IP	International Protection Symbol
	Keep dry

Using the Monitor

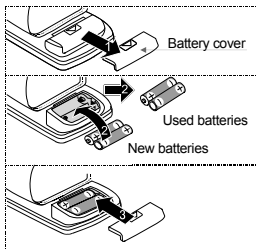Installing/Changing the Batteries

1. Remove the battery cover.
2. Remove the used batteries and insert new batteries into the battery compartment as shown, taking care that the polarities (+ and -) are correct. Use only LR03 or AAA batteries.
3. Attach the battery cover.

CAUTIONS

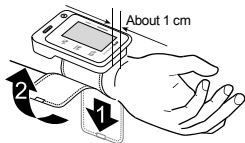
- Insert the batteries as shown in the battery compartment. If installed incorrectly, the device will not work.
- When  (LOW BATTERY mark) blinks on the display, replace all batteries with new ones. Do not mix old and new batteries. It may shorten the battery life, or cause the device to malfunction.
-  (LOW BATTERY mark) does not appear when the batteries are drained.
- The battery life varies with the ambient temperature and may be shorter at low temperatures. Generally, two new LR03/AAA batteries will last approximately for four months when used twice for measurement each day.
- Use the specified batteries only. The batteries provided with the device are for testing monitor performance and may have a limited life.
- Remove the batteries if the device is not to be used for a long period of time. The batteries may leak and cause a malfunction.

Using the Monitor

Applying the Cuff

1. Wrap the cuff around your wrist about 1 cm above your hand as shown in the figure at the right.
2. Apply the cuff tightly using the Velcro strip.

Note: For accurate measurements, apply the cuff tightly and measure on a bare wrist

How to Take Accurate Measurements

- Remain still and keep quiet during measurement.
- Sit down in a comfortable position. Place your elbow on a table with your palm facing upward and the cuff is at the same level as your heart.
- Relax for about five to ten minutes before measurement. If you are excited or depressed by emotional stress, the measurement will reflect this stress as a higher (or lower) than normal blood pressure reading and the pulse reading will usually be faster than normal.
- Try to measure your blood pressure at about the same time every day.
- An individual's blood pressure varies constantly, depending on what they are doing, what they have eaten and what they drink can have a very strong and rapid effect on your blood pressure.
- Do not measure immediately after physical exercise or a bath. Rest for twenty or thirty minutes before taking the measurement.
- Do not cross your legs. Keep your feet flat on the floor and straighten your back.
- This device bases its measurements on the heartbeat. If you have a very weak or irregular heartbeat, the device may have difficulty determining your blood pressure.

Using the Monitor

- Should the device detect a condition that is abnormal, it will stop the measurement and display an error symbol.
See page 6 for the description of symbols.
- This blood pressure monitor is intended for use by adults. Consult with your physician before using this device on a child. A child should not use this device unattended.
- The automatic blood pressure monitor's performance may be affected by excessive temperature or humidity, or altitude.

Measurement

- During measurement, it is normal for the cuff to feel very tight.

After Measurement

- While readings are displayed, if you press the START button to turn the device off, new readings are stored in memory.
- While readings are displayed, if you press the ◀ button to turn the device off, new readings are not stored.
- Remove the cuff and record your data.

Notes: The device is provided with an automatic power shut-off function which stores the current data in the memory and turns the device off automatically one minute after measurement. Allow at least three minutes between measurements on the same person.

Measurements

Note: The UB-525, once used, will provide an inflation appropriate to the user.

1. Wrap the cuff around your wrist. Sit comfortably with the cuff at the same level as your heart and relax.
2. Press the START button. All of the display segments are displayed.
3. Zero (0) is displayed blinking briefly. Then the display changes, as measurement begins. The cuff starts to inflate. It is normal for the cuff to feel very tight. The measurement starts automatically when inflation starts, and the ♥ (heart mark) blinks.

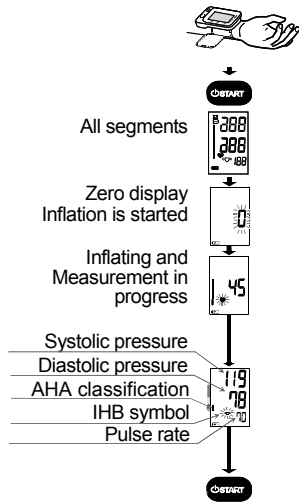
Note: If you wish to stop inflation at any time, press the START ◀ button again.

4. When the measurement is complete, the device displays readings (of the systolic and diastolic pressure readings, pulse rate, AHA classification, IHB symbol). The cuff exhausts the remaining air and completely deflates automatically.

Note: If you do not want to store new readings in memory, press the button while readings are displayed.

5. Press the START button again to turn the device off. Remove the cuff.


Notes: The device is provided with an automatic power shut-off function. Allow at least three minutes between measurements on the same person.

Recalling the Memory Data

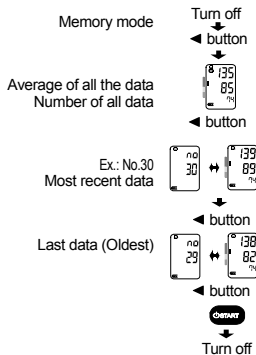Note: This device stores the last 60 measurements in memory.

1. Press the ◀ button when turning the device off. The average of all measurements and the number of data are displayed. If no data, “0” is displayed. Press the ◀ or START button to turn the device off.
2. Use the following buttons to display data (of number and measurement data).
 - The device displays the average of all measurements and the number of data are displayed.
 - Pressing the ◀ button each time , the device displays as follows:
 - Average data of all measurements taken between.
 - Data (of number and measurement data). The device displays in order from most recent data.In the example: No.30 & data → No.29 & data → ... → No.01 & data.
3. If you press the ◀ button after oldest data is displayed, the device proceeds to step 1, the average of all measurements and the number of data are displayed.
4. Press the START button to turn the device off. After one minute of non-operation, the device will turn off automatically.

Deleting Data Stored in Memory

When the device is turned off, press and hold the ◀ button. The device displays the  mark, deletes data stored in memory when the  mark is blinking and turns off automatically.

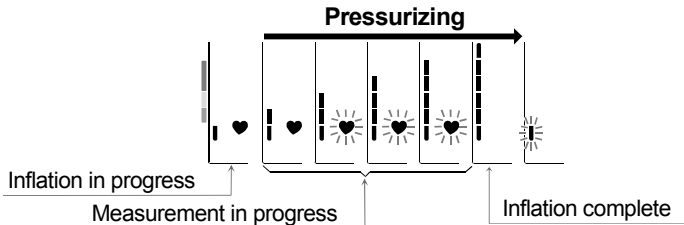
What is Irregular Heartbeat?

An irregular heartbeat is defined as a heartbeat that varies $\pm 15\%$ from the average of all heartbeats. When the monitor detects an irregular rhythm during the measurements, the IHB indicator will appear on the display with the measurement values.

Note: We recommend contacting your physician if you see this «» IHB indicator frequently.

Pressure Bar Indicator

The indicator monitors the progress of pressure during measurement.

AHA Classification Indicator

Each segment of the bar indicator corresponds to the AHA blood pressure classification.

Example

Hypertension Stage 2	Hypertension Stage 1	Elevated
		

AHA Classification Indicator

-  ← Hypertension Stage 2
-  ← Hypertension Stage 1
-  ← Elevated
-  ← Normal

- : The indicator displays a segment, based on the current data, corresponding to the AHA classification.

Blood Pressure Classification	Systolic (mmHg)		Diastolic (mmHg)
Normal	< 120	and	< 80
Elevated	120 – 129	and	< 80
Hypertension Stage 1 (High Blood Pressure)	130 – 139	or	80 – 89
Hypertension Stage 2 (High Blood Pressure)	≥ 140	or	≥ 90
HYPERTENSIVE CRISIS (consult your doctor immediately)	≥ 180	and / or	≥ 120

About Blood Pressure

What is Blood Pressure?

Blood pressure is the force exerted by blood against the walls of the arteries. Systolic pressure occurs when the heart contracts. Diastolic pressure occurs when the heart expands. Blood pressure is measured in millimeters of mercury (mmHg). One's natural blood pressure is represented by the fundamental pressure, which is measured first thing in the morning while one is still at rest and before eating.

What is Hypertension and How is it Controlled?

Hypertension, an abnormally high arterial blood pressure, if left unattended can cause many health problems including stroke and heart attack. Hypertension can be controlled by altering lifestyle, avoiding stress, and with medication under a doctor's supervision. To prevent hypertension or keep it under control:

- Do not smoke
- Exercise regularly
- Reduce salt and fat intake
- Have regular physical checkups
- Maintain proper weight

Why Measure Blood Pressure at Home?

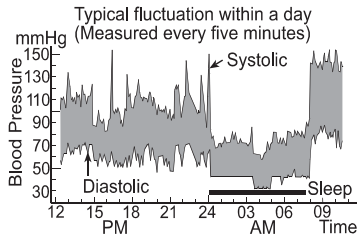
Blood pressure measured at a clinic or doctor's office may cause apprehension and can produce an elevated reading, 25 to 30 mmHg higher than that measured at home. Home measurement reduces the effects of outside influences on blood pressure readings, supplements the doctor's readings and provides a more accurate, complete blood pressure history.

About Blood Pressure

Blood Pressure Variations

An individual's blood pressure varies greatly on a daily and seasonal basis. It may vary by 30 to 50 mmHg due to various conditions during the day. In hypertensive individuals variations are even more pronounced. Normally, the blood pressure rises while at work or play and falls to its lowest levels during sleep. So, do not be overly concerned by the results of one measurement.

Take measurements at the same time every day using the procedure described in this manual to get to know your normal blood pressure. Regular readings give a more comprehensive blood pressure history. Be sure to note the date and time when recording your blood pressure. Consult your doctor to interpret your blood pressure data.

Troubleshooting

Problem	Possible Reason	Recommended Action
Nothing appears in the display, even when the device is turned on.	Batteries are drained.	Replace all batteries with new ones.
	Battery terminals are not in the correct position.	Reinstall the batteries with terminals matching those indicated in the battery compartment.
The cuff does not inflate.	Battery voltage is too low.  (Low Battery mark) blinks. If the batteries are drained completely, the mark does not appear.	Replace all batteries with new ones.
The device does not measure. Readings are too high or too low.	The cuff is not applied properly.	Apply the cuff correctly.
	You moved your wrist or body during the measurement.	Make sure you remain very still and quiet during the measurement.
	The cuff position is not correct.	Sit comfortably and still. Raise your hand so that the cuff is level at the same level as your heart.
	_____	If you have a weak or irregular heart beat, the device may have difficulty in determining your blood pressure.
Other	The value is different from that measured at a clinic or doctor's office.	See "Why Measure Blood Pressure at Home?"
	_____	Remove the batteries. Place them back properly and try measurement again.

Note: If the actions described above do not solve the problem, contact the dealer. Do not attempt to open or repair this product yourself, as any attempt to do so will make your warranty invalid.

Maintenance

Do not open the device. It uses delicate electrical components and an intricate air unit that could be damaged. If you cannot fix the problem using the troubleshooting instructions, contact our customer service department.

Technical Data

Type	UB-525
Measurement method	Oscillometric measurement
Measurement range	Pressure: 0 – 299 mmHg Systolic pressure: 60 – 279 mmHg Diastolic pressure: 40 – 200 mmHg Pulse: 40 – 180 beats / minute
Measurement accuracy	Pressure: ± 3 mmHg Pulse: $\pm 5\%$
Power supply	2 x 1.5V alkaline batteries (LR03 or AAA)
Number of measurements	Approx. 250 measurements, when AAA alkaline batteries are used, with pressure value of 170 mmHg at room temperature of 23 °C.
Wrist circumference	13.5 – 21.5 cm (5.3" – 8.5")
Classification	Internally powered ME equipment (Continuous operation mode)
Applied part	Cuff Type BF 
Useful life	Device: 5 years (when used six times a day)

Technical Data

Clinical test	According to ISO81060-2:2013
EMC	IEC 60601-1-2: 2014
Memory	Last 60 measurements
Operating conditions	10°C to 40°C / 15% RH to 85% RH / 800 hPa to 1060 hPa
Transport / storage conditions	-20°C to 60°C / 10% RH to 95% RH / 700 hPa to 1060 hPa
Dimensions	Approx. 56 [W] x 88 [H] x 21.5 [D] mm
Weight	Approx. 90 g, excluding the batteries
Ingress protection	IP20

Note: Specifications are subject to change for improvement without prior notice. IP classification is the degrees of protection provided by enclosures in accordance with IEC 60529. This device is protected against solid foreign objects of 12 mm diameter and greater such as a finger. This device is not protected against water.

Warranty

LIMITED WARRANTY

A&D Medical

For purchasers within the US only:

Product	Consumer Warranty Term
UB-525	5 year

For outside of US, please contact local distributor or dealer.

Limited Warranty:

A&D Medical (“A&D”) warrants to the first purchaser (“You”) that the A&D product You purchased (the “Product”) will be free from defects in material, workmanship and design for the applicable Warranty Term stated above from the date You purchased the Product under normal use. This Limited Warranty is personal to You and is not transferable. If the Product is defective, then You return the Product to A&D in accordance with the procedure set forth below. A&D’s warranty obligation is limited to the repair or replacement, at A&D’s option, of the defective Product that has been returned by You within the warranty period. Such repair or replacement will be at no charge to You. The repaired or replacement Product is warranted here-under for the longer of the remainder of the original warranty period or 90 days from the date of shipment of the repaired or replacement Product. If you return the Product for warranty service to A&D, You must return the Product, freight and insurance prepaid, within the warranty period to the address set forth be-low, together with satisfactory proof of the date of Your purchase and a description of the defect. Returns will not be accepted unless a Return Material Authorization (RMA) Number has been issued from A&D Customer Service Representative.

Warranty

In US

A&D Medical
Attn.: Warranty Department
1756 Automation Parkway
San Jose, CA 95131, U.S.A
1-800-726-3364

In Canada

Auto Control Medical
Attn.: Warranty Department
6695 Millcreek Dr., Unit 6
Mississauga, ON, L5N 5R, Canada
1-800-461-0991

This Limited Warranty does not cover, and A&D will not be liable for (i) any shipment damage, (ii) any damage or defect due to misuse, abuse, failure to use reasonable care, failure to follow written instructions enclosed with the Product, accident, subjecting the Product to any voltage other than the specified voltage, improper environmental conditions, or modification, alteration or repair by anyone other than A&D or persons authorized by A&D, or (iii) expendable or consumable components.

THIS LIMITED WARRANTY IS THE ONLY WARRANTY PROVIDED BY A&D; THERE ARE NO OTHER EXPRESS WARRANTIES. If A&D cannot reasonably repair or replace the Product, A&D will refund the amount You paid for the Product (not including taxes), less a reasonable charge for usage. To receive a refund you must have returned the Product and all associated materials to A&D. The above remedy of repair, replacement or refund is your only and exclusive remedy. IN NO EVENT SHALL A&D BE LIABLE FOR ANY DAMAGES, INCLUDING, WITHOUT LIMITATION, LOST PROFITS, LOST INFORMATION OR REPLACEMENT COSTS, ARISING OUT OF YOUR USE OF OR INABILITY TO USE THE PRODUCT, INCLUDING, WITHOUT LIMITATION, ANY SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES, EVEN IF A&D HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. Some states do not allow the exclusion of incidental or consequential damages, so that the above exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that may vary from state to state.

No distributor, dealer or other party is authorized to make any warranty on behalf of A&D or to modify this warranty, or to assume for A&D any liability with respect to its products.

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A&D Company, Limited
A&D Engineering, Inc.
San Jose, CA 95131 U.S.A.