

Deluxe Connected Weight Scale UC-352CNBLE

1) Install the included batteries into your scale.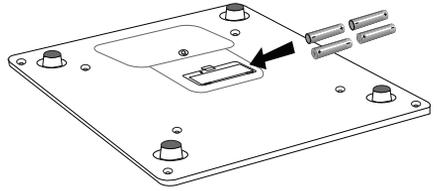

2) To change the units, push and release the set switch. Default is lb.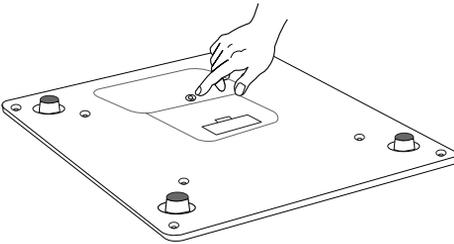


3) Download the free "A&D Connect" app from the App Store or Google play.


4) Turn on *Bluetooth*[®] settings on your mobile device (smartphone or tablet).

5) Install and launch the “A&D Connect” app. The app will guide you through the account set up and connection process.

iOS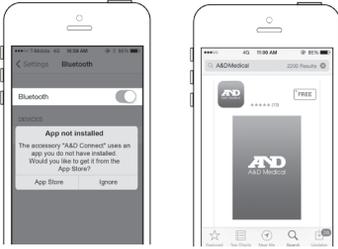


Android

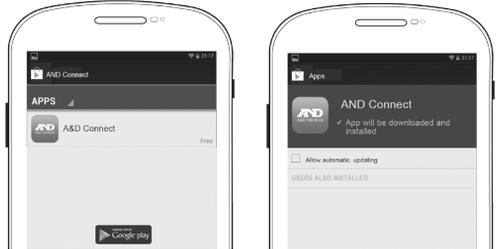
The app will show you the following:


6) Press and hold the set button to start connection.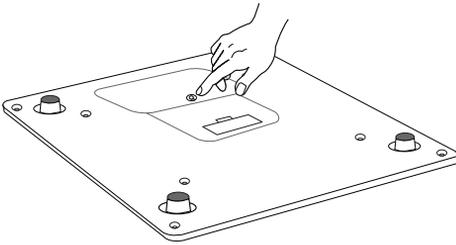


7) Release the set switch when “Pr” is displayed.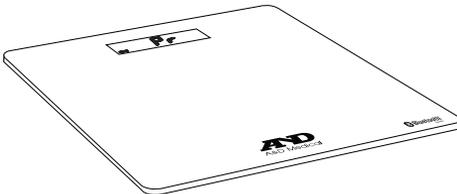

8) When "Pr" and the  symbol are displayed, the scale is pairing with your mobile device.

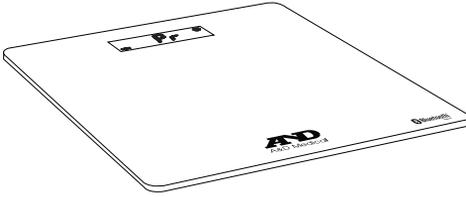
9) The scale displays "End" to indicate pairing is complete.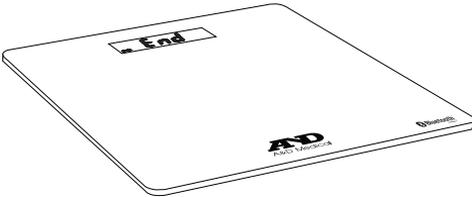

10) The scale is now ready to be used.


11) Step on the scale to take a reading.

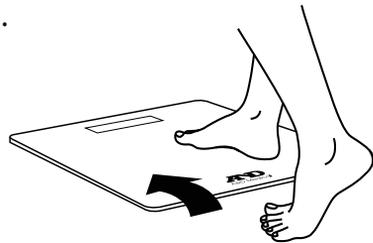12) Stay on the scale until "STEP" and "off" are alternately displayed.

13) Step off the scale.

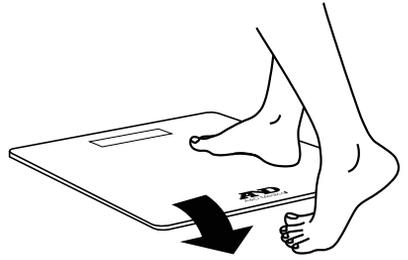14) See your weight. Measurements are sent to your mobile device. When the A&D Connect app is open and running. Results are stored in the scale when your mobile device is not available.

AND 1-800-726-9966

A&D Medical www.andmedical.com

©A&D Medical. All rights reserved. iPad, iPhone and iPod are trademarks of Apple Inc. Android and Google play are trademarks of Google Inc.

Specifications subject to change without notice. UC-352CNBLE-QSG-Eng-02/2015.

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