

WIRELESS BLOOD PRESSURE MONITOR UA-651BLE

ENG

1 Install the included batteries into your blood pressure monitor.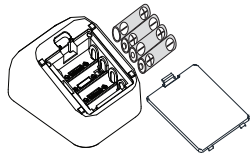

2 Insert the air connector plug into the air socket.

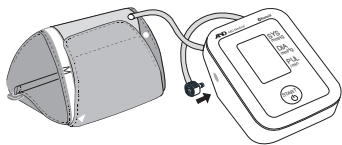
3 Turn on *Bluetooth*® settings on your mobile device (smartphone or tablet).

4 Download the free **A&D Heart Track** app from the App Store or Google play.


5 Getting started with A&D Heart Track app.


5a. Set up account. Create a username and password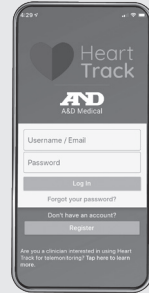


5b. Select Menu

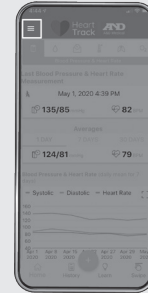5c. Click Add Bluetooth Devices

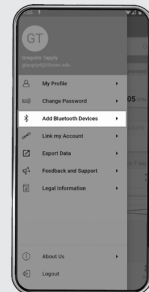
5d. Click + and select blood pressure monitor

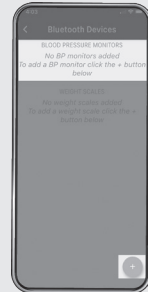
5e. Select UA-651BLE


5f. Complete the pairing process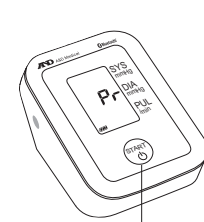


Press and hold the START button

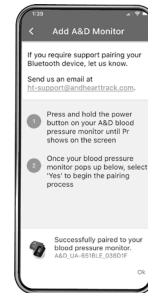

5g. The blood pressure monitor displays “End” to indicate the pairing process is complete.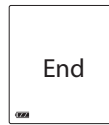


5h. Click + at the bottom

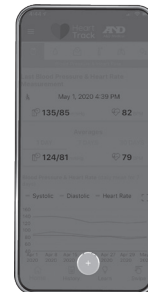
5i. Select Blood Pressure & Heart Rate. Select Auto Bluetooth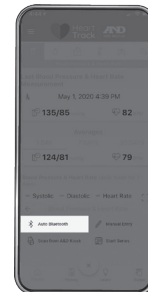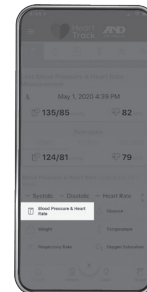

6 Lay your left arm flat on the table with palm up. Place the cuff on your upper arm.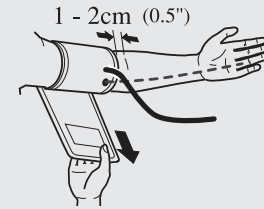

7 Fasten the cuff tight enough so you can slide two fingers underneath it.

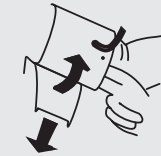8 Sit comfortably with your arm resting on a flat surface, elevated to heart level, palm up.

9 Press the START button.

10 Sit quietly as cuff inflates and deflates.

11 When the measurement is complete, the cuff deflates automatically.

12 Your systolic and diastolic pressure and pulse rate appear on the screen. Measurements are sent to your mobile device when the A&D Heart Track app is open and running. Results are stored in the memory when your mobile device is not available.

Systolic

Diastolic

Pulse

TIPS FOR TAKING BLOOD PRESSURE

BEFORE YOUR MEASUREMENT:

- For 30 minutes prior to taking your blood pressure
 - » Do not exercise
 - » Do not drink coffee, caffeinated soda or alcohol
 - » Do not smoke
- Sit quietly for 5-10 minutes
- Ensure you are using the right size cuff

DURING YOUR MEASUREMENT:

- Do not talk
- Sit with your back straight and supported
- Uncross your legs and place feet flat on floor
- Rest arm on a table so that the cuff is at heart level
- Measure two times a day, in the morning and evening

1-888-726-9966 | andmedical.com

© 2022 A&D Medical. All rights reserved. Specifications subject to change without notice. iPad, iPhone and iPod are trademarks of Apple, Inc. Android and Google play are trademarks of Google, Inc. Specifications subject to change without notice. UA-651BLE-QSG

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