PREMIUM WIRELESS
BLOOD PRESSURE MONITOR
UPPER ARM
UA-1200CNBLE

Instruction Manual

GREETINGS

This A&D blood pressure monitor is one of the most advanced monitors available and is designed for ease of use and accuracy. This device will facilitate your daily blood pressure regimen.

We recommend that you read through this manual carefully before using the device for the first time.

The *Bluetooth*[®] word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by A&D is under license. Other trademarks and trade names are those of their respective owners.

INTENDED USE

UA-1200CNBLE UltraConnect wireless blood pressure monitor is designed for adult at home or clinic use. It measures systolic, diastolic, and pulse rate. It uses the oscillometric method to determine blood pressure. If the user has a massive body movement or irregular heartbeats, a heart symbol will be shown on the app display. The device has *Bluetooth*[®] low energy wireless (BLE) communication capability with Smart devices. It fits arm size from 22 to 42 cm.

PRELIMINARY REMARKS

- This device complies with part 15 of the FCC rules and contains the FCC ID VPYLBZY.
- This device is designed for use on adults only, and is not intended for infants.
- This device is designed for use in the home healthcare environment to measure blood pressure and pulse rate of people for diagnosis.

PRECAUTIONS

- Precision components are used in the construction of this device. Extremes in temperature, humidity, direct sunlight, shock or dust should be avoided.
- Clean the device and cuff with a dry, soft cloth or a cloth dampened with water and a neutral detergent. Never use alcohol, benzene, thinner or other harsh chemicals to clean the device or cuff.
- The device and cuff are not water resistant. Prevent rain, sweat and water from soiling the device and cuff.
- Measurements may be distorted if the device is used close to televisions, microwave ovens, X-ray or other devices with strong electrical fields.
- When using the device, confirm that it is clean.
- Used equipment, parts and batteries are not treated as ordinary household waste, and must be disposed of according to the applicable local regulations.
- Do not modify the device. It may cause accidents or damage to the device.

PRECAUTIONS

- To measure blood pressure, the arm must be squeezed by the cuff hard enough to temporarily stop blood flow through the artery. This may cause pain, numbness or a temporary red mark to the arm. This condition will appear especially when measurement is repeated successively. Any pain, numbness, or red marks will disappear with time.
- If you feel strong pain during the measurement, please remove the cuff.
- Measuring blood pressure too frequently may cause harm due to blood flow interference.
- Do not apply the cuff on a arm in which another medical device is attached. The equipment may not function properly.
- People who have a severe circulatory deficit in the arm must consult a doctor before using the device, to avoid medical problems.
- Do not self-diagnose the measurement results and start treatment by yourself. Always consult your doctor for evaluation of the results and treatment.
- Do not apply the cuff on a arm with an unhealed wound.
- Do not apply the cuff on a arm receiving an intravenous drip or blood transfusion. It may cause injury.
- Do not use the device where flammable gases, such as anesthetic gases, are present. It may cause an explosion.
- Do not use the device in highly concentrated oxygen environments, such as a high-pressure oxygen chamber or an oxygen tent. It may cause a fire or explosion.


PRECAUTIONS

- This device complies with Part 15 of FCC rules. Operation is subject to the following two conditions:
 - (1) This device may not cause harmful interference, and
 - (2) This device must accept any interference received, including interference that may cause undesired operations.
- When the AC adapter is used, make sure that it can be readily removed from the electrical outlet when necessary.
- Sometimes, the temperature of the cuff will be about 50°C. If you feel hot, please remove the cuff.
- Please do not simultaneous use HF surgical equipment.
- Clinical testing has not conducted on pregnant women.
- If you have had a mastectomy, please consult a doctor before using the device.
- Do not let children use the device by themselves and do not leave the device within the reach of children.
- There are small parts that may cause a choking hazard if swallowed by mistake by infants.
- Do not touch the batteries and the patient at the same time. That may result in electrical shock.
- Wireless communication devices, such as home networking devices, mobile phones, cordless phones and their base stations, walkie-talkies can affect this blood pressure monitor. Therefore, a minimum distance of 3.3 meters/10.8 feet should be kept from such devices.

TABLE OF CONTENTS

Parts Identification	E-7
Contents	E-7
Symbols	E-8
Using the Monitor	E-11
Measurement	E-17
What is an Irregular Heartbeat	E-21
Pressure Bar Indicator	E-22
Settings	E-23
Troubleshooting	E-25
Maintenance	E-26
Technical Data	E-27
Contact Information	E-29
Warranty	E-30

PARTS IDENTIFICATION

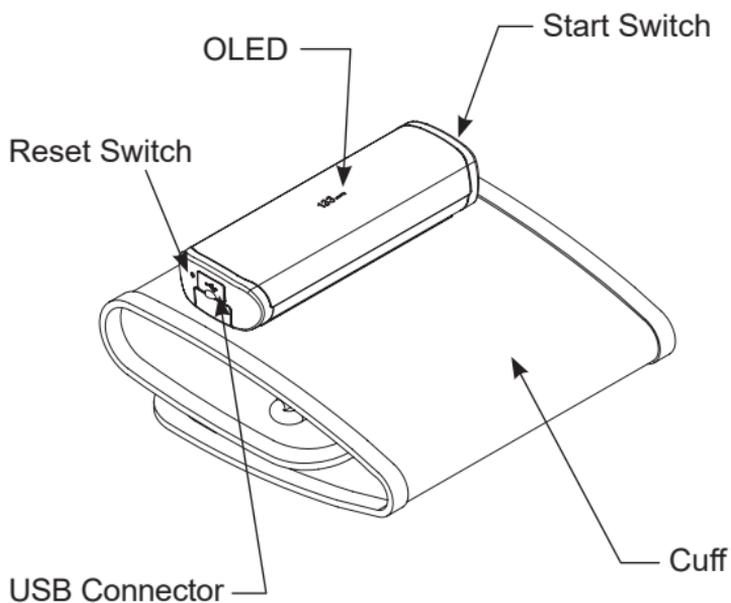CONTENTS

- UA-1200CNBLE UltraConnect Blood Pressure Monitor
- 1 Instruction Manual
- 1 Quick Start Guide
- 1 Charging Cable

SYMBOLS

Symbols that appear on the display

Symbols	Function/Meaning	Recommended Action
	A&D logo	—
SYS mmHg	Systolic blood pressure in mmHg	—
DIA mmHg	Diastolic blood pressure in mmHg	—
PUL /min	Pulse per minute	—
	The battery power is low	Charge the device
	User Select mode	Select the user
	Application is connected with the device	—
	Application has disconnected from the device.	—
	Appears while measurement is in progress.	Measurement is in progress. Remain as still as possible.
^{PUL} E /min	The pulse is not detected correctly.	Apply the cuff correctly, and take another measurement.
Err 1	Unstable blood pressure due to movement during measurement. The systolic and diastolic values are within 10 mmHg of each other.	Take another measurement. Remain still during measurement.

SYMBOLS

Symbols	Function/Meaning	Recommended Action
Err 3	The pressure value did not increase during inflation. The cuff is not applied correctly.	Apply the cuff correctly, and take another measurement.
Err 9	Blood pressure monitor internal error.	Press the reset button. After reset, clock data will disappear. If the error still appears, contact A&D Medical.
	Data is transferring	—
	Airplane mode	Measurements can be taken but device will not transmit data wirelessly.
	Appears when a body or arm movement is detected.	The reading may yield an incorrect value. Take another measurement. Remain still during measurement.
	Appears during measurement when the cuff is attached loosely.	The reading may yield an incorrect value. Apply the cuff correctly, and take another measurement.
	Firmware is updating	—

SYMBOLS

Symbols printed on the device case and cuff

Symbols	Function/Meaning
	Stand by and turn the device on.
	Direct current
	Type BF: Device and cuff are designed to provide special protection against electrical shocks.
SN	Serial number
	Refer to instruction manual/booklet
	Keep dry
BT	<i>Bluetooth</i> [®] address
IP	Ingress Protection symbol

USING THE MONITOR

The UA-1200CNBLE UltraConnect Wireless Blood Pressure Monitor features 4 measurement modes.

1. In-App Mode Measurement:

User can take measurements and view their trends in the A&D Heart Track app.

2. Offline Mode Measurement:

Measurement can also be completed without the A&D Heart Track app. Data will be stored on the monitor and transferred to the app on next connection.

3. Multiuser Mode Measurement:

Up to 6 users (5 accounts and 1 visitor) can use and store data on the app. Each user must have a mobile device paired with the monitor.

4. Visitor Mode Measurement (Offline):

Select “0 Visitor” to take a measurement in visitor mode from the monitor only. Data is not stored in visitor mode.

USING THE MONITOR

Charging the Device

Battery may be partially charged but be sure to charge the device completely upon first use after purchase.

1. Remove the USB connector cover.
2. Connect the charging cable.
3. During charging, the red LED will illuminate.
4. When charging is complete, the red LED will turn off.
5. Once charging is complete, remove the charging cable.

CAUTION!

- The battery life varies with the ambient temperature and may be shorter at low temperatures.
- Please use the specified charging cable shipped with the device.
- Blood pressure measurement cannot be done during charging.
- When disconnecting the AC adapter from the electrical outlet, grasp and pull the AC adapter body out of the outlet.
- When disconnecting the charging cable from the blood pressure monitor, grasp and pull the charging cable out of the outlet.
- Do not connect to equipment other than those specified.
- Do not change the battery.
- If the battery can no longer be charged, please contact the dealer.
- Overcharging the battery may reduce its lifetime.
- Lithium battery replacement by inadequately trained personnel could result in a hazard such as a fire or explosion.

USING THE MONITOR

Wireless Function

- In the unlikely event that this monitor causes radio wave interference to a different wireless station, change the location where this monitor is used or stop use immediately.
- Be sure to use in a location where visibility between the two devices that you want to connect is good. The connection distance is reduced by the structure of buildings or other obstructions. In particular, connection may be impossible when devices are used on either side of reinforced concrete.
- Do not use the *Bluetooth*[®] connection in the range of a wireless LAN or other wireless devices, near devices that emit radio waves such as microwaves, in locations where there are many obstructions, or in other locations where signal strength is weak. Doing so may result in frequent loss of connection, very slow communication speeds and errors.
- Using close to an IEEE802.11g/b/n wireless LAN device may cause mutual interference to occur, which may result in reduced communication speeds or which may prevent connection. In this case, switch off the power supply to the device that is not being used, or use the monitor in a different location.
- If the monitor does not connect normally when used near a wireless station or broadcast station, use the monitor in a different location.
- A&D Medical cannot accept liability for any damages incurred due to impaired operation or data loss, etc. that occur through the use of this product.
- This product is not guaranteed to connect to all *Bluetooth*[®] compatible devices.

USING THE MONITOR

Bluetooth® Transmission **Bluetooth®**

This product is equipped with a *Bluetooth®* 4.1 wireless function and can connect to the following *Bluetooth®* devices.

Apple iOS v.8.0 -v.11.0
Selected iOS devices only

Android OS 4.4.4 - 8.0
Bluetooth® device.
Selected Android phones only.

To connect with your mobile device – download and install the “A&D Heart Track” app onto your mobile device. Follow the pairing instructions in the quick start guide. Please ensure Airplane Mode is turned OFF. Check page E-24

Monitor Pairing

1. Ensure the *Bluetooth®* function of the mobile device is ON.
2. Open the A&D Heart Track.
3. Click on the menu icon in upper left.
4. From the menu, select “Add Bluetooth Devices.”
5. Tap the “+” icon and choose “Blood Pressure Monitor.”
6. Select the UA-1200CNBLE monitor and follow the on screen instructions.

USING THE MONITOR

Multiuser Pairing

The monitor can be registered for up to 6 users. Follow the above instructions to pair the mobile device for each user.

New users can be added anytime. The user who has the lowest frequency of measurement times will be cleared once the device has reached the 6 user max.

Note: If new user uses the same mobile device, then re-pairing is not required as this has been complete.

CAUTION!


The communication distance between this monitor and the receiver device is about 33 feet (10m). This distance can be reduced by the conditions in the surrounding environment, so be sure to check that the distance is short enough for a connection to be made after measurement is complete.

USING THE MONITOR

Applying the Arm Cuff

Wrap the cuff around the upper arm, about 0.5" (1-2cm) above the inside of the elbow, as shown. Sit comfortably to ensure proper fit/orientation on the arm.

Note: White tab should be closest to the palm.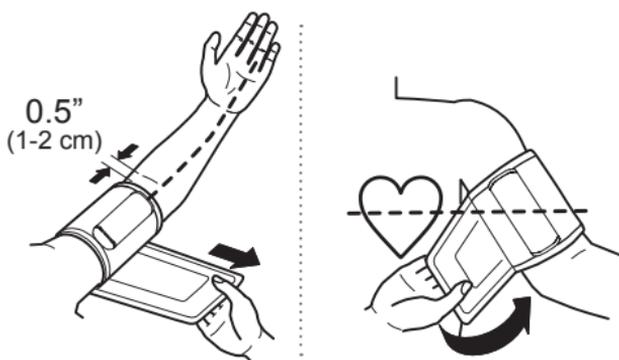

MEASUREMENT

In-App Mode Measurement

1. Ensure the *Bluetooth*[®] function of the mobile device is ON.
2. Open the A&D Heart Track.
3. Tap the “+” or “+ Measure” button on the home screen.
4. Follow the on screen instructions.
5. The cuff starts to inflate. It is normal for the cuff to feel tight. The app displays the pressure value once inflation starts.
6. When the measurement is complete, the app and the monitor will display results (*Systolic, diastolic and pulse rate will scroll on the monitor*).

Note: Press the start button again to turn the monitor off. The monitor is also provided with an automatic power shut-off function. Allow at least one minute between measurements on the same person.

MEASUREMENT

Offline Mode Measurement


1. Press the **Start** button on the monitor.

After the A&D logo appears for 1 second, the User Select screen will display.

2. Press the **Start** button again to select a user or switch between username from User Select screen.

Username can be changed in the app under Settings. When this is not set, the first name will display as the username.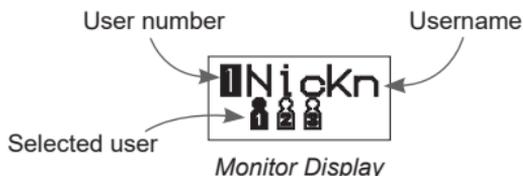

3. Measurement starts automatically after 3 seconds.

4. Zero(0) is displayed blinking briefly.

Then the display changes, as measurement begins.

5. The cuff starts to inflate. It is normal for the cuff to feel tight. The monitor displays the pressure value once inflation starts.

6. When the measurement is complete, the monitor will display results similar to the icon on the right. (*Systolic, diastolic and pulse rate will scroll.*)

Note: Press the start button again to turn the monitor off. The monitor is also provided with an automatic power shut-off function. Allow at least one minute between measurements on the same person.

MEASUREMENT

Multiuser Mode Measurement

1. Up to 6 people can use and store readings on the app. Each user must have a mobile device paired with the monitor. (See *pg. E-15 for pairing instructions.*)
2. If you are taking the reading without the app, press the **Start** button on the monitor to switch between usernames. Select the desired user before each reading. Data will transfer to the app on your next connection.

Note: Usernames will be displayed in order of greater usage frequency.

Visitor Mode Measurement

1. Press the **Start** button on the monitor.
2. Press the **Start** button again to switch between usernames and select “0 Visitor”.
3. Measurement starts automatically after 3 seconds.
4. When the measurement is complete, the monitor will display results similar to the icon on the right. (*Systolic, diastolic and pulse rate will scroll.*)

Visitor Mode is only available through the monitor. Data is not stored in visitor mode.

Tips for a Good Measurement

- Sit comfortably on a chair with your feet on the floor and your back straight. Do not cross your legs.
- Place your arm on a table with your palm facing upward and the cuff at the same level as your heart.
- Relax for about five to ten minutes before measurement.
- Remain still and keep quiet during measurement.
- Do not smoke, exercise, or consume anything for at least 30 minutes beforehand.
- This device bases its measurements on the heartbeat. If you have a very weak or irregular heartbeat, the device may have difficulty determining your blood pressure.
- Should the device detect a condition that is abnormal, it will stop the measurement and display an error symbol. Refer to the section “Symbols” for the description of the symbols.
- Try to measure your blood pressure at the same time every day.
- The automatic blood pressure monitor’s performance may be affected by excessive temperature or humidity, or altitude.

134 ^{SYS} mmHg

WHAT IS AN IRREGULAR HEARTBEAT

An irregular heartbeat is defined as a hearbeat that varies from the average of all heartbeats. When the monitor detects an irregular rhythm during the measurements, the IHB indicator will appear on the display with the measurement values.


Note: We recommend contacting your physician if you see this symbol «» frequently.

PRESSURE BAR INDICATOR

The pressure category indicator is a feature which provides a snapshot of your blood pressure classification based on your measurements. This will let you quickly know what your blood pressure readings mean. Each segment of the bar indicator corresponds to the AHA blood pressure classification.

Pressure Bar Indicator

The indicator highlights a segment, based on the current measurement, corresponding to the AHA Classification.

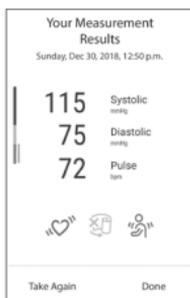App Display

AHA Classification Table			
BP Classification	Systolic (mmHg)		Diastolic
Normal	less than 120	and	less than 80
Elevated	120-129	and	less than 80
Stage 1 Hypertension	130-139	or	80-89
Stage 2 Hypertension	more than 140	or	more than 90
Hypertensive Crisis	more than 180	and/or	more than 120

SETTINGS

Setting	Description
Nickname	User can overwrite the default monitor display username and assign a personal username (up to 10 characters).
Start Time	Enables a user to choose how long of an interval between pressing “Start” button and when the monitor will start measurement. For multiple users, a longer interval is useful. For single users, a shorter interval is useful. Default for single user is set at 1 second and for multiple users at 5 seconds.
Remove User	Allows user to remove himself as a user in the BP monitor.
Check Software Update	Enables users to update the monitor with the most recent <i>Bluetooth</i> [®] software.
Unpair	Allows user to unpair his mobile device with the BP monitor. This can also be accomplished via the <i>Bluetooth</i> [®] setting in the mobile device.
Reset	This function will reset the monitor to factory settings. All data in the monitor will be deleted. This will not affect any data stored in your mobile device.

SETTINGS

Monitor Setting

The airplane mode feature can be used in an environment that does not allow to emit radio frequency. Measurement can be done in airplane mode and data will transfer to the app on next connection.

1. Hold the **Start** button and the airplane mode select screen will display. Tap the **Start** button to switch between ON and OFF.

2. Select ON and wait 3 seconds to enter airplane mode.

In airplane mode the airplane icon to the left will be illuminated.

TROUBLESHOOTING

Symptom	Probable Cause	Treatment
Power does not turn on.	Low Battery.	Charge the battery.
	Blood Pressure monitor internal error.	Press the Reset Switch.
	Battery charging cable is connected.	Remove the charging cable.
The cuff does not inflate.	Low Battery.	Charge the battery.
The monitor is not connecting to application	Airplane mode is ON	Check page E-24 to turn Airplane mode OFF
	Blood Pressure monitor internal error.	Press the Reset Switch.
The device does not measure.	You moved your arm or body during measurement.	Make sure you remain still and quiet during measurement.
Readings are too high or too low.	—	If you have a very weak or irregular heartbeat, the device may have difficulty in determining your blood pressure.
	The cuff position is not correct.	Sit comfortably and still. Place your arm on a table with your palm facing upward and the cuff at the same level as your heart.

TROUBLESHOOTING

Symptom	Probable Cause	Treatment
Other	The value is different from that measured at a clinic or doctor's office.	At a clinic or doctor's office apprehension may cause an elevated reading. Home measurement reduces the effects of outside influences on blood pressure readings & supplementing the doctor's readings.

Note: If the actions described above do not solve the problem, contact the dealer. Do not attempt to open or repair this product, as any attempt to do so will make your warranty invalid.

MAINTENANCE

Do not open the device. It uses delicate electrical components and an intricate air unit that could be damaged. If you cannot fix the problem using the troubleshooting instructions, contact customer service. Customer service can provide technical assistance and spare parts.

TECHNICAL DATA

Type	UA-1200CNBLE
Measurement method	Oscillometric measurement
Measurement range	Pressure: 0 - 299 mmHg Systolic pressure: 60 - 279 mmHg Diastolic pressure: 40 - 200 mmHg Pulse: 40 - 180 beats/min
Measurement accuracy	Pressure: ± 3 mmHg Pulse: ± 5 %
Arm circumference	22 - 42cm
Power supply	3.7V Li-ion 325mAh
Number of measurements	Approx. 100 times With pressure value 170 mmHg, room temperature 23 °C.
Classification	Internally powered ME equipment <i>(Supplied by batteries)/</i> Class II <i>(Supplied by adapter)</i> Continuous Operation mode
EMC	IEC60601-1-2:2014
Clinical test	According to ISO81060-2
Wireless communication	ZY (MURATA Manufacturing Co.Ltd.) Bluetooth® Ver. 4.1 Low Energy Technology BLP Frequency band: 2402 MHz to 2480 MHz Modulation: GFSK Maximum RF output power: 2.1dBm
Operating conditions	+10 to +40 °C / 15 to 85% RH / 800 to 1060 hPa
Transport/Storage conditions	-20 to +60 °C / 10 to 95% RH / 700 to 1060 hPa

TECHNICAL DATA

Dimensions	Approx. 44 [W] x 126 [H] x 26 [D] mm
Weight	Approx. 205g
Ingress protection	Device: IP20
Applied part	Cuff Type BF
Useful life	5 years (<i>when used six times a day</i>)
Included items	UA-1200CNBLE - 1PC Instruction Manual - 1PC Charging cable (KSUSB-2C) - 1PC Quick Guide - 1PC

Industry Canada Caution

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This transmitter must not be co-located or operated in conjunction with any other antenna or transmitter.

Note: This equipment has been tested and found to comply with Industry Canada rules as a Bluetooth device modular approval spread/spectrum/digital device (2400-2483.5 MHz under IC ID IC 772C-LBZY).

These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and **receiver**.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.

TECHNICAL DATA

•Consult the dealer or an experienced radio/TV technician for help

Bluetooth® devices carry the Bluetooth® logo mark.

iPhone, iPad and iPod are trademarks of Apple Inc., registered in the U.S. and other countries. Google Play and Google Play logo are registered trademarks of Google, Inc.

Warranty

LIMITED WARRANTY

A&D Medical ("A&D")

For purchasers within the US only:

Product	Consumer Warranty Term
UA-1200CNBLE	5 Years

Limited Warranty:

A&D Medical ("A&D") warrants to the first purchaser ("You") that the A&D product You purchased (the "Product") will be free from defects in material, workmanship and design for the applicable Warranty Term stated above from the date You purchased the Product under normal use. This Limited Warranty is personal to You and is not transferable. If the Product is defective, then You return the Product to A&D in accordance with the procedure set forth below. A&D's warranty obligation is limited to the repair or replacement, at A&D's option, of the defective Product that has been returned by You within the warranty period. Such repair or replacement will be at no charge to You. The repaired or replacement Product is warranted here-under for the longer of the remainder of the original warranty period or 90 days from the date of shipment of the repaired or replacement Product. To obtain a warranty service, please contact us in **US at 1-888-726-9966 or in Canada at 1-800-461-0991** for return address, shipping and handling fee, and other instructions for processing warranty. Please ensure you have satisfactory proof of the date of Your purchase and a description of the defect.

Warranty

Returns will not be accepted unless a Return Material Authorization (RMA) Number has been issued from A&D Customer Service Representative.

This Limited Warranty does not cover, and A&D will not be liable for (i) any shipment damage, (ii) any damage or defect due to misuse, abuse, failure to use reasonable care, failure to follow written instructions enclosed with the Product, accident, subjecting the Product to any voltage other than the specified voltage, improper environmental conditions, or modification, alteration or repair by anyone other than A&D or persons authorized by A&D, or (iii) expendable or consumable components.

THIS LIMITED WARRANTY IS THE ONLY WARRANTY PROVIDED BY A&D; THERE ARE NO OTHER EXPRESS WARRANTIES. If A&D cannot reasonably repair or replace the Product, A&D will refund the amount You paid for the Product (not including taxes), less a reasonable charge for usage. To receive a refund you must have returned the Product and all associated materials to A&D. The above remedy of repair, replacement or refund is your only and exclusive remedy. IN NO EVENT SHALL A&D BE LIABLE FOR ANY DAMAGES, INCLUDING, WITHOUT LIMITATION, LOST PROFITS, LOST INFORMATION OR REPLACEMENT COSTS, ARISING OUT OF YOUR USE OF OR INABILITY TO USE THE PRODUCT, INCLUDING, WITHOUT LIMITATION, ANY SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES, EVEN IF A&D HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. Some states do not allow the exclusion of incidental or

Warranty

consequential damages, so that the above exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that may vary from state to state.

No distributor, dealer or other party is authorized to make any warranty on behalf of A&D or to modify this warranty, or to assume for A&D any liability with respect to its products.

PREMIUM WIRELESS BLOOD PRESSURE MONITOR

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A&D Company, Limited
A&D Engineering, Inc.