PREMIUM EAR THERMOMETER UT-302

Instruction Manual
ORIGINAL

Greetings

Congratulations on your purchase of the A&D Medical UT-302 Instant-Read Ear Thermometer. This easy to use thermometer offers a safe, accurate and quick way to take a temperature measurement through the ear. It is an especially convenient method of taking a child's temperature and perfect for adults too.

Before Use

- In order to obtain an accurate measurement, please read this instruction manual carefully before using.

Precautions

- This thermometer is designed to take a temperature through the ear canal. DO NOT use it to take rectal, oral or underarm temperature.
- This ear thermometer is a delicate electronic device. Please handle with care.
- Only the tip of the ear thermometer is waterproof. Follow the proper cleaning instructions as outlined in this manual. Do not submerge body of thermometer in water or any other liquid.
- Do not expose thermometer to direct sunlight.
- Precision components were used in the construction of this device. Extremes in temperature, humidity, shock or dust should be avoided.
- Keep this device out of the reach of children.
- Do not self-diagnose your condition using the measured result. Consult your doctor if your temperature is higher than the normal temperature.
- Do not use the ear thermometer if there is blood or drainage in the external ear canal.
- Do not use the ear thermometer in an infected ear.
- Do not force the thermometer tip into the ear canal.
- Do not disassemble the ear thermometer.

Contents

Parts Identification	E-3
Symbols	E-4
Taking Your Temperature	E-5
Switching The Unit Of Measure	E-10
Replacing the Battery.....	E-11
Care and Cleaning	E-12
Troubleshooting	E-13
Technical Data	E-14
Warranty.....	E-15

Parts Identification

Probe

Use this part to insert into your ear

Power & Scan button

Press this button to power on and scan the ear temperature

LCD Display


Display the measurement results

Fronthead Cover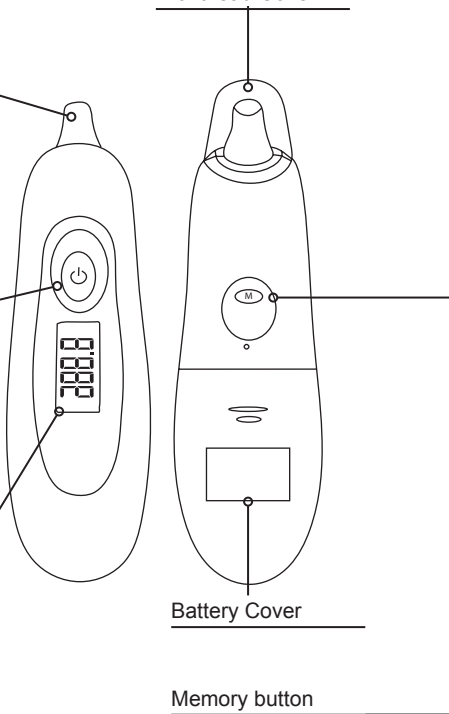

Battery Cover

Memory button

Press this button to track historical data

Symbols

	Degrees of temperature
	Scanning sequence in progress
	Smiling face (indicates temperature is lower than 38°C or 100.4°F)
	Sad face (indicates temperature is higher than 38°C or 100.4°F)
	Celsius Scale
	Fahrenheit Scale
	Low battery warning
	Memory symbol

Taking your Temperature

Before You Start

This product is intended for household use only. If you have questions about the meaning of a specific temperature, please consult your physician.

Use the thermometer at room temperature between 60.8°F - 104.0°F (16°C - 40°C). If the ear thermometer is stored in an environment where the temperature is out of that range, move the thermometer to a location with a room temperature of 60.8°F - 104.0°F (16°C - 40°C). Allow the thermometer to equalize to that temperature range before use.

DO NOT measure temperature if:

- Your ear is cold, wet or blocked with ear wax
- It is within 30 minutes after eating, taking a bath or physical exercise
- You are suffering from an ear infection

NOTE: Make sure the tip of the thermometer is clean. A dirty lens may cause an inaccurate measurement. Clean the lens with a soft, alcohol moistened cloth or cotton swab. Never use an abrasive cleaner. Never submerge the body of the thermometer in water or any other liquid.

We recommend you practice with the A&D Medical ear thermometer on yourself and family members while you are healthy. This way you know how the thermometer works and will feel more confident of the measurements when you or a family member is ill.

Taking your temperature


How The Ear Thermometer Works

The A&D Medical ear thermometer utilizes infrared thermo scanning principles to scan the temperature of the human eardrum. The eardrum is located beneath the surface and not easily influenced by the changing temperature in the environment.

The A&D Medical ear thermometer is faster, safer and easier to use than traditional stick thermometers. It scans the eardrum 512 times per second and displays the highest temperature detected. Measurements can even be taken while your child is sleeping. For adults, an ear thermometer offers fast, comfortable, accurate readings in seconds.


Why The Ear?

Research indicates that the ear is an ideal site for taking body temperature. The eardrum shares blood vessels with the hypothalamus, the part of the brain that controls body temperature. Therefore, the ear is an accurate indicator of internal body temperature.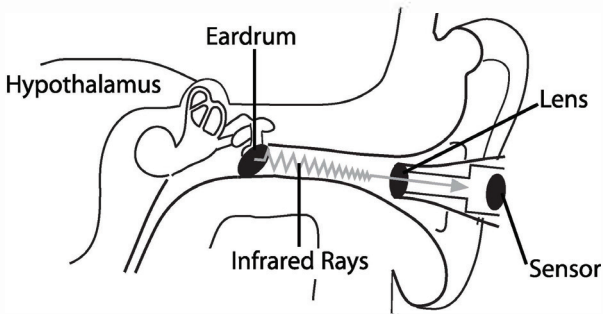

Taking Your Temperature


1. Press Fronthead cover's sides to remove.

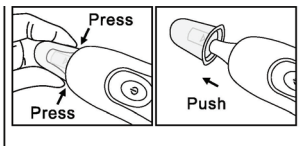
2. Press the Power/Scan button for one second consecutively. The system will perform a self-test, and all symbols will be displayed on the LCD for one second

3. After the self-test, the last temperature taken will be shown on the display. The UT-302 is now ready to take a new measurement

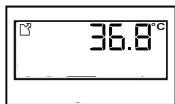
4. Press the Power/Scan "⏻" again. The symbol "◀■■■" will appear beneath the last reading.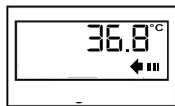

Taking Your Temperature


5. Insert the probe into the ear canal. The probe must be inserted well into ear to allow the probe tip to continuously sense the ear canal. This will ensure an accurate temperature measurement.

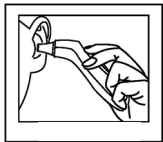
6. After one second a beep will sound indicating the reading is complete. Remove the thermometer from the ear to get the temperature reading.

7. A smiling face "☺" will appear indicating a temperature reading lower than 38°C (100.4°F).

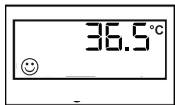
A sad face "☹" accompanied by one long and two short beeps will indicate a temperature of 38°C (100.4°F) or higher.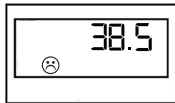


Taking Your Temperature


8. A "Lo" or a "Hi" symbol appears when the temperature measured is out of the measurement range.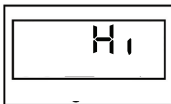

9. The symbol "Err" appears if the operating temperature is not within 16°C (60.8°F) and 35°C (95°F).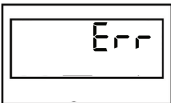



10. To recall past readings, press the "M" button on the underside of the unit. The memory symbol and the set index appear at the top left side of the display.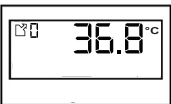

11. To insure a long battery life, this thermometer will automatically power off after one minute of non-use.

Switching The Unit Of Measure


1. To switch between °C or °F, press the Power/Scan "⏻" button once. Then press and hold the Power/Scan "⏻" button. Now, press the "M" button to toggle between the °C or °F modes.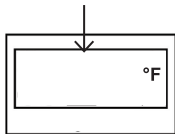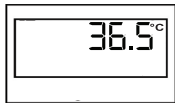

Replacing the Battery

Low Battery Warning


Replace the battery when the low battery warning symbol "⎓" appears on the display. When the low battery symbol appears, the thermometer is still fully functional. When the "Lo" and "⎓" appear on the display you should stop using the thermometer until the battery has been replaced.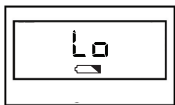

Battery Installation

1. Using a small phillips head screwdriver, remove the screw from the battery lid and remove the lid.
2. Using a non-metal pointed tool, carefully dislodge the battery and remove.
3. Insert a new 3 Volt CR-2032 Lithium battery, with the positive (+) side facing up.
4. Place the lid back on and screw in to secure.


Note: Battery removal clears readings stored in memory.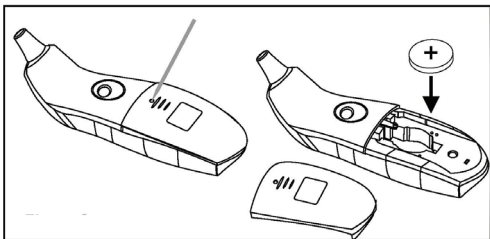

Care and Cleaning

Probe Lens

Clean the lens of the thermometer with a soft, alcohol moistened cotton swab after each use. Do not use water to wash the probe lens.

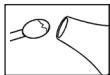Thermometer

Clean the body of the thermometer with a soft, dry cloth - DO NOT use water, cleaning solutions or any other liquid. Never use an abrasive cleaner or submerge the body of the thermometer in water or any other liquids.

Troubleshooting

Problem	Recommended Action
Why do I get consistently low temperature readings?	The thermometer is not positioned properly in the ear. The tip of the thermometer must form a seal around the opening of the ear.
	The waterproof lens is dirty. Clean the lens with a soft, alcohol moistened cotton swab.
Why is the reading taken in the right ear different from that taken in the left ear?	The temperature is roughly the same in both ears, but the values may differ because the shape and cleanliness of each ear is different. Also, the position of the thermometer can affect the reading.
Can I use this thermometer to measure an animal's temperature?	No. This thermometer has been designed exclusively for human use. The UT-302 ear thermometer was programmed specifically for the human ear construction and human body temperature.

Technical Data

Model	UT-302
Dimensions	2.2" x 1.75" x 6.2" 55.3 X 44.4 X 158.5mm
Weight	2.3 oz. (65.5 g)
Sensor type	Thermopile Sensor STI 18
Measuring accuracy	±0.4°F (0.2°C): From 96.8°F to 102.2°F (36°C - 39°C) ±0.5°F (0.3°C): Less than 96.8°F (36°C) or greater than 102.2°F (39°C)
Measuring range	50°F - 122°F (10°C - 50°C)
Display resolution	0.1 °F or °C
Response time	1.0 Second
Memory function	10 memory recall
Thermometer tip	Equipped with a waterproof lens. No lens cover is required.
Automatic shut down	The thermometer will automatically turn off after one minute of non-use
Power supply	One 3 Volt CR-2032 Lithium battery
Battery life	Approximately 5,000 measurements
Operating environment	60.8°F - 95°F (16°C - 35°C) with relative humidity up to 95%
Storage environment	-4°F - 122°F (-20°C - 50°C) with relative humidity up to 95%

Warranty

LIMITED WARRANTY

A&D Medical

For purchasers within the US and Canada only:

Product	Consumer Warranty Term
UT-302	2 Years

For outside of US and Canada, please contact local distributor or dealer.

Limited Warranty:

A&D Medical (“A&D”) warrants to the first purchaser (“You”) that the A&D product You purchased (the “Product”) will be free from defects in material, workmanship and design for the applicable Warranty Term stated above from the date You purchased the Product under normal use. This Limited Warranty is personal to You and is not transferable. If the Product is defective, then You return the Product to A&D in accordance with the procedure set forth below. A&D’s warranty obligation is limited to the repair or replacement, at A&D’s option, of the defective Product that has been returned by You within the warranty period. Such repair or replacement will be at no charge to You. The repaired or replacement Product is warranted here-under for the longer of the remainder of the original warranty period or 90 days from the date of shipment of the repaired or replacement Product. If you return the Product for warranty service to A&D, You must return the Product, freight and insurance prepaid, within the warranty period to the address set forth below, together with satisfactory proof of the date of Your purchase and a description of the defect.

Returns will not be accepted unless a Return Material Authorization (RMA) Number has been issued from A&D Customer Service Representative.

In US

A&D Medical

Attn.: Warranty Department

4622 Runway Boulevard

Ann Arbor, MI 48108 USA

1-800-726-3364

In Canada

Auto Control Medical

Attn.: Warranty Department

6695 Millcreek Dr., Unit 6

Mississauga, ON, L5N 5R, Canada

1-800-461-0991

Warranty

This Limited Warranty does not cover, and A&D will not be liable for (i) any shipment damage, (ii) any damage or defect due to misuse, abuse, failure to use reasonable care, failure to follow written instructions enclosed with the Product, accident, subjecting the Product to any voltage other than the specified voltage, improper environmental conditions, or modification, alteration or repair by anyone other than A&D or persons authorized by A&D, or (iii) expendable or consumable components.

THIS LIMITED WARRANTY IS THE ONLY WARRANTY PROVIDED A&D; THERE ARE NO OTHER EXPRESS WARRANTIES. If A&D cannot reasonably repair or replace the Product, A&D will refund the amount You paid for the Product (not including taxes), less a reasonable charge for usage. To receive a refund you must have returned the Product and all associated materials to A&D. The above remedy of repair, replacement or refund is your only and exclusive remedy. IN NO EVENT SHALL A&D BE LIABLE FOR ANY DAMAGES, INCLUDING, WITHOUT LIMITATION, LOST PROFITS, LOST INFORMATION OR REPLACEMENT COSTS, ARISING OUT OF YOUR USE OF OR INABILITY TO USE THE PRODUCT, INCLUDING, WITHOUT LIMITATION, ANY SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES, EVEN IF A&D HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. Some states do not allow the exclusion of incidental or consequential damages, so that the above exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that may vary from state to state.

No distributor, dealer or other party is authorized to make any warranty on behalf of A&D or to modify this warranty, or to assume for A&D any liability with respect to its products.

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A&D Company, Limited
A&D Engineering, Inc.
San Jose, CA 95131 U.S.A.