

TM-2440

*Recorder for Ambulatory Blood
Pressure Monitor*

INSTRUCTION MANUAL

Ambulatory Blood Pressure Monitor

Español



Français



A&D
A&D Medical

© 2022 A&D Company, Limited. All rights reserved.

- No part of this publication may be reproduced, transmitted, transcribed, or translated into any language in any form by any means without the written permission of A&D Company, Limited.
- The contents of this manual and the specifications for the instrument covered by this manual are subject to change for improvement without notice.
- Other trademarks and trade names are those of their respective owners.

Compliance

Compliance with the Australian EMD Framework

The device conforms to the following requirements:




EMD Emission standard for industrial, Scientific & Medical equipment AS/NZS 2064:1997, EMD Generic Immunity standard AS/NZS 4252.1:1994.

Warning Definitions







To prevent accidents due to inappropriate handling, this product and its manual contain the following warning signs and marks.

The meanings of these warning signs and marks are as follows.

Warning Definitions

 Danger	An imminently hazardous situation that will result in death or serious injury, if not avoided.
 Warning	A potentially hazardous situation that could result in death or serious injury, if not avoided.
 Caution	A potentially hazardous situation that may result in minor or moderate injury, if not avoided. It may also be used to warn against unsafe practice.

Symbol Examples

	The symbol  indicates "Caution". The nature of the caution required is described inside or near the symbol, using text or a picture. The example indicates caution against electrical shock.
	The symbol  indicates "Do not". The prohibited action is described inside or near the symbol, using text or a picture. The example indicates "Do not disassemble".
	The symbol  indicates mandatory action. The mandatory action is described inside or near the symbol, using text or a picture. The example indicates general mandatory action.

Other

Note	Provides information useful for the user when operating the device.
-------------	---

Precautions for each operation are described in the pages of this manual. Read the instruction manual before using the device.

Precautions for Use

In order to use the TM-2440 (the recorder for the ambulatory blood pressure monitor) safely and correctly, read the following precautions carefully before using the monitor. The following content summarizes general matters affecting the safety of patients and operators, as well as safe handling of the monitor. Precautions for each operation are described in the pages of this manual. Read the instruction manual before using the device.

1. Precautions When Wearing and Storing the Recorder.

Danger



Keep the recorder away from areas where flammable anesthetics or flammable gases are present, high-pressure oxygen chambers and oxygen tents. Using the recorder in these areas may cause an explosion.

Do not use the recorder together with a magnetic resonance imaging system (MRI).

Caution



To preserve the capabilities of the device, consider the following environmental conditions when using and storing the recorder. The performance of the recorder may be affected by excessive temperature, humidity and altitude.

- Avoid locations where the recorder may be splashed by water.
- Avoid locations with high temperature, high humidity, direct sunlight, dust, salt and sulfur in the air.
- Avoid locations where the recorder may be tilted, vibrated, or impacted (including during transportation).
- Avoid locations where chemicals are stored or gas is present.

Caution



- Operation conditions:
Temperature: +10 °C to +40 °C,
Humidity: 30 %RH to 85 %RH (no condensation).
- Transport and storage conditions:
Temperature: -20 °C to +60 °C,
Humidity: 10 %RH to 95 %RH (no condensation).

2. Precautions Before Using the Recorder.

Caution



- Confirm that the recorder is operating safely and correctly.
- When the recorder is used in conjunction with other devices, it may cause an incorrect diagnosis or safety problems.
Confirm that the devices can be connected safely.
- Check for mutual interference with other medical devices.
Confirm that the recorder can be used correctly.
- Use accessories, options and consumables specified by A&D.
- Carefully read the instruction manuals provided with optional items. Any cautions and warnings are not described in this manual.
- For safe and correct use of the recorder, inspect it before use.
- Leave the recorder in a normal operation state for one hour or more before use and turn it on.



- Only connect **dedicated peripherals** to the USB connector.
Do not connect other devices.
- Do not connect any cuff except for a cuff authorized by A&D to the air socket.

Note

Preparation of the Recorder

- Delete the last data stored in the recorder before it is used by the next patient.
- Replace the batteries before the recorder is used by the next patient.

Device


- Use the recorder for diagnosis and countermeasures only.
- Confirm that the air hose and cuff are worn correctly. (Example: kinks and tension in the air hose, position and direction of the cuff)



Instructions for the Patient Wearing the Device

- Inform the patient how to suspend automatic blood pressure measurement to stop the recorder if a problem occurs when alone.
- Inform the patient to remove the recorder quickly when in pain or if any problem occurs.
- Take care when using around babies and infants, as there is a danger of accidental suffocation with the air hose.



3. Precautions for Batteries Used for Blood Pressure Measurement.




Caution



- Install the batteries in accordance with polarity signs "+" and "-" shown on the inside of the battery cover. (Note the polarities)
- Replace flat batteries with new ones at the same time.
- Remove the batteries if the recorder is not going to be used for a long period of time. The battery may leak and cause a malfunction.
-  □ Use two alkaline batteries (size AA) or specified re-chargeable batteries (size AA, Ni-MH).
- Push and hold the "-" spring terminal with the battery. Slide and install the "+" terminal of the battery at the "+" terminal of the battery compartment. If the battery is installed from the "+" terminal, the battery cover may be damaged.

	<ul style="list-style-type: none"> ❑ Do not touch the battery and the patient at the same time. It may cause an electric shock.
	<p>Do not mix old batteries with new ones. Do not use batteries of different type and manufacturer. If these are used, this may cause leakage, heat and explosion. The recorder may malfunction.</p>

4. Precautions During Use.

 Danger	
	<p>Do not use the recorder while operating automobiles or other vehicles.</p> <p>Example: The recorder may inhibit the motion of the body or arms when operating the vehicle. etc.</p>

 Warning	
	<p>This medical device can be only operated by a doctor or a legally authorized person. Explain the correct usage to the patient and ensure they can stop measurement when a problem occurs.</p>
	<p>Do not use a mobile phone near the recorder (less than 30 cm). It may cause a malfunction.</p>

 Caution	
	<ul style="list-style-type: none"> ❑ Stop using the recorder and suspend automatic blood pressure measurement if the patient feels pain in their arm or the measurement is incorrect. ❑ Do not use the recorder in a strong magnetic or electric field. ❑ Do not use the recorder on a patient using a heart-lung machine.

Note

Instructions for Patient

If the temperature is low, battery power becomes lower and the measurement count is reduced.

5. Precautions After Using the Recorder.

Caution

Processing work of Measurement Data

Be sure to process measurement data immediately using a **dedicated peripheral**.

The Recorder



- After cleaning the accessories, arrange and store them.
- Clean the recorder for the next measurement.
- Suspend automatic blood pressure measurement. Otherwise, pressurization for automatic measurement is started at the next measurement start time and the cuff or other parts may be broken by the inflation.
- Remove the batteries from the recorder if it is not going to be used for a long period of time. The batteries may leak and break the recorder.
- Avoid having a child use the recorder on their own.
Do not put the recorder in a place within reach of an infant.
Doing so may cause accidents or damage.



Hold the connector housing when connecting and removing the cable. Do not pull on the cable.

Note



Precautions After Using the Recorder (TM-2440)

Be sure to process measurement data immediately using a **dedicated peripheral** after finishing measurement.

Backup Lithium Rechargeable Battery

The recorder comes with a backup lithium battery. This battery supplies power to the built-in clock when replacing AA batteries used for blood pressure measurement. The lithium battery is charged from the AA batteries.

How to Extend the Life of the Backup Battery

- When first using after purchase or after storing for a month or more, replace the batteries and charge the backup battery. It is enough if the backup battery is charged for 48 hours or more.
(The backup battery is always charged by the AA batteries.)
- Replace with two new AA batteries when the battery indicator displays .
- When  is displayed on the battery indicator, the blood pressure measurement and data communication cannot be performed. Replace with two new AA batteries.
- Remove the batteries to prevent the recorder from liquid leakage from a battery if the recorder is not used for a month or more.

6. Remedies When the Device Has an Error

Warning



- Stop the operation and remove the AA batteries. If the battery terminals are shorted, the battery may be hot.
- In the event of a failure, the recorder may get hot during measurement, please handle it with care.
- Put the **"Malfunction" "Do not use"** notice label on the recorder. Contact your dealer.
- Stop the recorder immediately when the measurement time is above 180 seconds and the air pressure rises above 299 mmHg.

7. Precautions for Maintenance

Warning



- Confirm the correct performance and safety of the recorder when it has not been used for a long period of time.
- To maintain correct measurement and safety, perform an inspection and maintenance before use. The user (hospital, clinic, etc.) is responsible for the management of the medical equipment. If the inspection and maintenance are not performed correctly, an accident may occur.

Warning



- Use a dry lint free cloth to clean the recorder.
- Do not use volatile agents, such as a thinner or benzine.
- Do not use a wet cloth.



- Do not disassemble or modify the recorder (medical electronic device). It may cause damage.

8. Precautions and Remedies for a Malfunction Due to a Strong Electromagnetic Field

Caution



- The recorder complies with EMD-standard IEC60601-1-2: 2014. However, to prevent electromagnetic interference with other devices, do not use mobile phones near the recorder.
- If the recorder is located near strong electromagnetic fields, noise may affect the waveforms and malfunctions may occur. If an unexpected malfunction occurs during use, check for electromagnetic interference and take appropriate actions.

Caution



The following examples are general causes of malfunctions and their remedies.

- Use of mobile phones
Radio waves may cause unexpected malfunctions.
 - Wireless communication devices, home networking devices such as cordless phones and similar communication devices can affect the recorder. Therefore, they must be kept at a distance of at least 30 cm or more from the recorder.
- If there is static electricity in the area of use (discharges from devices or the surrounding area)
 - Before using the recorder, ensure that the operator and patient have discharged any static electricity.
 - Humidify the room.

9. Environmental Protection

Caution



Before disposing of the recorder, remove the lithium battery from the recorder.

Precautions For Safe Measurement

This section describes precautions for the measurement and the sensor. Notify the patient of following content and explain it to them. Guide the patient in the correct use of the device.

Blood Pressure Measurement

Warning



Ensure the tube is not bent excessively and that air flows properly. If a bent air hose is used, air pressure may remain in the cuff, which may stop blood flow to the arm.





- ❑ Do not measure the blood pressure on an arm if the patient has the following conditions. This may cause an accident or aggravation of the injury.
 - 1) An injury or disease on an arm.
 - 2) An arm receiving an intravenous drip or blood transfusion.
 - 3) A limb that is shunted for artificial dialysis.
 - 4) The patient has been bedridden for a long time (Where there is a possibility of thrombus).

Caution



- ❑ Confirm the condition of the patient if there are measurement problems. The device guesses that the condition is worsening beyond the limit of measurement or if the air flow ceases because the air hose is bent.
- ❑ Measuring blood pressure too frequently may cause bodily harm due to blood flow interference. Confirm that the operation of the device does not result in prolonged impairment on blood circulation when using the device repeatedly.
- ❑ Blood pressure measurement may not be accurate if the patient has continuous arrhythmia, or moves excessively.

Caution


	<ul style="list-style-type: none">□ Wear the cuff at the same level as heart. (If the level is different, an error in the measurement value occurs.)□ The recorder responds to artifact and external impact. If there are any doubts in the measurement value, measure blood pressure by auscultation or palpation.□ A measurement error may occur if the cuff is not of the correct arm circumference for the patient.
	Do not inflate the cuff before it is wrapped around the arm of the patient. Damage and explosion of the cuff may occur.

Note

<ul style="list-style-type: none">□ Blood pressure measurement may cause subcutaneous bleeding. This subcutaneous bleeding is temporary and disappears with time.□ If the patient uses a heart-lung machine, blood pressure cannot be measured due to the absence of a heartbeat.□ Blood pressure cannot be measured correctly if thick clothing is worn.□ Blood pressure cannot be measured correctly if the clothing is rolled up and the arm is squeezed.□ Blood pressure cannot be measured correctly if peripheral circulation is insufficient, the blood pressure is excessively low or if the patient has hypothermia (blood flow is insufficient).□ Blood pressure cannot be measured correctly if the patient has frequent arrhythmia.□ Blood pressure cannot be measured correctly with the incorrect cuff size.□ Blood pressure cannot be measured correctly if the cuff is not worn at the same level as the heart.□ Blood pressure cannot be measured correctly if the patient is moving or talking during measurement.□ Clinical trials have not been conducted on newborn infants and pregnant women.□ Consult a doctor before use if you have had a mastectomy.


Cuff

Warning

- | | |
|---|--|
|  | <input type="checkbox"/> Dispose of cuffs contaminated by blood to prevent infectious disease from spreading. |
| | <input type="checkbox"/> Avoid storing the cuff folded or with a tightly twisted air hose for extended periods of time. Such treatment may shorten the life of the components. |

Measurement of Pulse Rate

Warning

- | | |
|---|--|
|  | Do not use the displayed pulse rate for the diagnosis of an irregular heartbeat. |
|---|--|

Note

The recorder measures the pulse rate when measuring the blood pressure.

Packing List

Caution




The recorder is a precision instrument. Use with caution.
Excessive shock may cause failure and malfunction.

Note


The recorder is shipped using a special packing box designed to protect it from damage during transport. When you open this box, make sure you have everything on the packing list. If you have any questions, contact your local dealer or the nearest A&D dealer. We recommend keeping the special packing box.

Refer to "**10. Optional Items (requiring order)**" for options.

Blood pressure recorder	1	
Accessories		
Adult cuff 20 to 31 cm (7.8" to 12.2") for left arm	1	
Large cuff 28 to 38 cm (11.0" to 15.0") for left arm	1	
Adult cuff cover	2	
Large cuff cover	2	
Carry holder	AX-133025995	1
Belt	AX-00U44189	1
Clip		1
Activity record sheet (10 sheets)	AX-PP181-S	1
USB cable	AX-KOUSB4C	1
This instruction manual		1
Data manager instruction simplified manual		1
Quick start guide		1

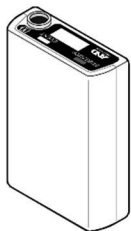


Blood pressure recorder

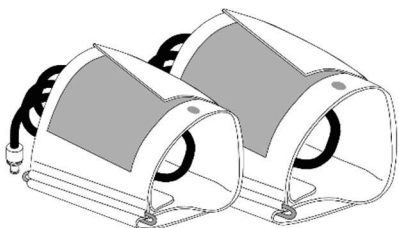

Adult cuff for left arm

Large cuff for left arm


Adult cuff cover × 2

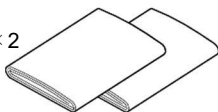


Carrying holder

Clip

Large cuff cover × 2

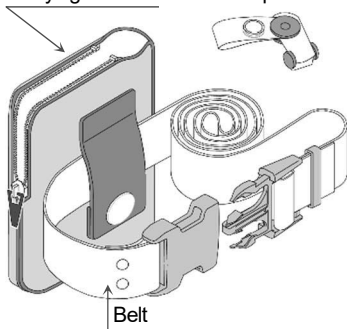


Belt

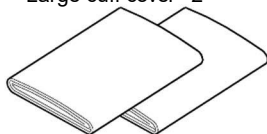


Activity record sheet (10 sheets)


Data manager instruction simplified manual

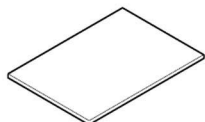

Quick start guide

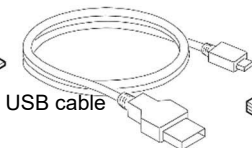

USB cable

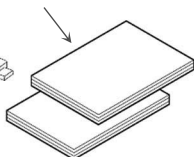

This instruction manual

Contents

Compliance.....	i
Compliance with the Australian EMD Framework	i
Warning Definitions	ii
Precautions for Use	iii
Precautions For Safe Measurement.....	xi
Blood Pressure Measurement	xi
Cuff	xiii
Measurement of Pulse Rate.....	xiii
Packing List.....	xiv
1. Introduction.....	4
2. Features	4
3. Abbreviations & Symbols	6
4. Specifications	10
4.1. Recorder	10
4.2. Dimensions.....	13
5. Component Names.....	14
5.1. Recorder	14
5.2. OLED Display (Organic light emitting diode)	15
5.3. Principal Switch Operations	16
5.3.1. A-BPM Operations	16
5.3.2. Other Operations	19
6. Blood Pressure Measurement Functions.....	20
6.1. Automatic Blood Pressure Measurement (A-BPM).....	20
6.1.1. A-BPM Waiting Mode.....	21
6.1.2. Sleep Function and Interval time.....	22

6.1.3.	Stopping Measurement.....	22
6.2.	Measurement Results	23
6.2.1.	Displaying Measurement Results	23
6.2.2.	Storing Measurement Results.....	23
6.2.3.	Outputting Measurement Results	24
6.2.4.	ID numbers.....	24
7.	Preparing the Recorder	25
7.1.	Installing the Batteries (Replacing Batteries)	25
7.1.1.	How to Replace Batteries	27
7.2.	Preparing the Carry Holder.....	27
7.3.	Inspection for Use	28
7.3.1.	Battery Pre-installation Checklists.....	28
7.3.2.	Battery Post-installation Checklists.....	29
8.	Operation.....	29
8.1.	Operation Flowchart	29
8.2.	Initial Settings.....	31
8.2.1.	Factory Settings.....	31
8.2.2.	The Clock and the Monitor Function of Measurement.....	32
8.2.3.	Initial Pressurization Value.....	33
8.3.	A-BPM Preset Programs	33
8.3.1.	A-BPM Items and Parameters	35
8.3.2.	A-BPM Program Examples	38
8.4.	Deleting Measurement Data.....	40
8.5.	Attaching the Product to the Patient.....	41
8.5.1.	Information for Patients	41
8.5.2.	Cuff Cover.....	44
8.5.3.	Attaching the Cuff, Carry holder and Recorder	45
8.6.	Blood Pressure Measurement Operations	49
8.6.1.	A-BPM Operations	49

8.6.2.	Manual Measurement	51
8.6.3.	Stopping and Suspending Measurements	52
8.7.	Connecting the Recorder to a Dedicated Peripheral	53
8.7.1.	Connecting with a USB cable.....	53
9.	Maintenance	55
9.1.	Product Storage, Inspection and Safety Management.....	55
9.2.	Cleaning the Product	56
9.3.	Periodic Inspection.....	58
9.3.1.	Battery Pre-installation Inspection.....	58
9.3.2.	Battery Post-installation Inspection.....	59
9.4.	Disposal.....	60
9.5.	Troubleshooting.....	61
9.6.	Error Codes	62
10.	Optional Items (requiring order)	65
11.	Appendix.....	67
11.1.	Principle of Blood Pressure Measurement	67
11.2.	EMD Information	69
12.	Limited Warranty.....	73

1. Introduction

Thank you for your Purchase!

The TM-2440 ambulatory blood pressure recorder enables accurate measurement of patient blood pressure automatically for preset times (e.g. 24-hours continuously). This manual explains the settings, operation, modes and programs for blood pressure measurement, as well as communication with a **dedicated peripheral**, maintenance, specifications and warnings. Read this manual for proper use and keep it in an accessible place.

2. Features

Summary

The recorder is an ambulatory blood pressure monitor that can non-invasively measure the blood pressure value and pulse rate of the patient under the guidance of a doctor. The purpose is to measure and store variation of the blood pressure over a day during daily life. The recorder is designed to be portable and have a data management function and simple operation.

Blood pressure measurement target

This recorder is designed for adults.

Purpose of use

The recorder enables automatic blood pressure measurement and manual blood pressure measurement. Blood pressure readings can be used for consulting with doctors and for self-managing health.

Automatic blood pressure measurement (A-BPM)

This mode can specify six pairs of arbitrary start times and intervals for every 24 hours and can automatically measure and record blood pressure.

Manual blood pressure measurement

Blood pressure can be measured manually at any time, including when the A-BPM function is activated.

Portability

The weight of the recorder is approximately 120 g (excluding batteries).

It is palm sized and equipped with a micro-pump.

Two AA alkaline batteries can be used. (LR6 or AA size)

Two rechargeable batteries (AA size, Ni-MH battery) can be used.

Operability

The settings of the recorder and the blood pressure measurement program can be configured easily using the ABPM Data Manager installed on the computer (**dedicated peripheral**).

Extensive analytical performance

The measurement interval time can be set for the automatic blood pressure measurement.

The blood pressure can be measured immediately using manual measurement at any time.

The analysis can be carried out effectively using the ABPM Data Manager installed on the computer (**dedicated peripheral**).

Shorter measurement time







The deflation speed is controlled to minimize the measurement time.














The pressurization value is controlled to minimize the measurement time.

Simple convenience

A **dedicated peripheral** can receive data over a USB cable. Data received can be analyzed and printed easily.

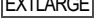3. Abbreviations & Symbols

Symbols	Meaning
SYS	Systolic blood pressure
DIA	Diastolic blood pressure
PUL	Pulse rate
PP	Pulse pressure $PP = SYS - DIA$
kPa mmHg	Unit of blood pressure
/min	Unit of pulse rate/minute
	Displaying: A-BPM is in operation.
	Memory full, delete data to start measurement.
	Battery indicator If the level 1  is displayed, blood pressure measurement and data communication cannot take place. Replace the batteries with 2 new LR6 (AA size) batteries.
	A-BPM sleep mark
	The mark is displayed during configuration.
Exx	Error codes. xx = 00 to 99
OLED	Organic light emitting diode

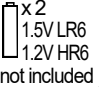
Symbols	Meaning
	Alert mark
	Degree of protection against electric shock: Equipment type BF.
	Manufacturer of the CE Marking. Date of manufacture.
	Symbol for small cuff Arm circumference 15 to 22 cm 5.9" to 8.7"
	Symbol for adult cuff Arm circumference 20 to 31 cm 7.8" to 12.2"
	Symbol for large cuff Arm circumference 28 to 38 cm 11.0" to 15.0"
	Symbol for extra-large cuff Arm circumference 36 to 50 cm 14.2" to 19.7"
	Symbol printed on packing. The adult cuff is included in the accessories.
	Refer to the instruction manual or booklet.
	Symbol for "Keep dry" and "Keep away from rain".
SN	Serial number
	Symbol printed in the battery compartment. Direction (polarity) to install battery.
	Symbol printed on packing. Batteries are excluded from accessories.
EMD	Electromagnetic disturbances
	Symbol for "Handle with care".

Symbols	Meaning
BPM	Blood pressure measurement
A-BPM	Automatic blood pressure measurement.
Sleep, Cycle, Hour, START, Operation	A-BPM symbols. #1
Not made with natural rubber latex.	Caution for patient. This is printed on the cuff.
<div style="border: 1px solid black; padding: 5px;"> <p style="text-align: center;">⚠ Caution</p> <ul style="list-style-type: none"> • Use alkaline batteries or specified rechargeable batteries and ensure correct polarity (+, -). • Do not mix new, used or different branded batteries. • Firmly secure cuff air hose to main body. </div>	<p>⚠ Cautions on battery cover.</p> <ul style="list-style-type: none"> □ Use alkaline batteries or the rechargeable batteries specified and ensure correct polarity (+, -). □ Do not mix new, used or different branded batteries. □ Firmly secure cuff air hose to main body.

#1: Refer to **"6.1. Automatic Blood Pressure Measurement (A-BPM)"** and **"8.3. A-BPM Preset Programs"** for 24-hours blood pressure recorder.

Waiting mode

The A-BPM **waiting mode** is a state where the blood pressure is not being measured during the **interval time**.

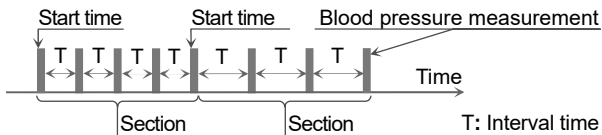

Dedicated peripheral

A **dedicated peripheral** means the computer that the ABPM Data Manager software is installed on. ABPM Data Manager can be downloaded by visiting


<https://www.aandd.jp/products/medical/professional/abpm.html>

Use a peripheral device that is compliant with the requirements for medical electrical equipment (IEC60601-1) when connecting the recorder to peripheral device. Do not connect the recorder to other device (Example: IEC60950) in the area using medical equipment. Use a USB cable shorter than 1.5 m (4.9 ft).

4. Specifications

4.1. Recorder

Items	Descriptions
Measurement method	Oscillometric measurement method
Pressure detection method	Semiconductor pressure sensor
Pressure display range	0 to 299 mmHg
Measurement accuracy	Pressure: ± 3 mmHg Pulse rate: ± 5 %
Minimum display division	Pressure: 1 mmHg Pulse rate: 1 beat/minute
Measurement range	Systolic pressure: 60 to 280 mmHg Diastolic pressure: 30 to 160 mmHg Pulse rate: 30 to 200 beats/minute
Depressurization	Constant exhaust with a controlled leakage valve for safety
Exhaust	Electromagnetic valve
Pressurization method	Micro-pump
Automatic pressurization	85 to 299 mmHg
Interval time (of A-BPM)	Intervals at each section which divides 24 hours into six parts at the maximum. Interval: OFF, 5, 10, 15, 20, 30, 60, 120 minutes
Clock	24-hour clock
Display	OLED, 96 x 39 pixels, white characters
Memory	Measurement data: 600 data points max.


Items	Descriptions
Power supply	With the same type of batteries: <ul style="list-style-type: none"> □ 2 x 1.5V batteries (LR6 or AA size) □ Alkaline battery or nickel-hydrate battery (Ni-MH) 1900 mAh or more Backup battery for built-in clock: Lithium rechargeable coin cell battery ML2016H
Measurement count	200 times or more. (when new alkaline batteries or nickel-hydrate batteries are used. It may vary due to measurement conditions.)
Rated voltage	DC 3.0 V (Alkaline battery, LR6), DC 2.4 V (Nickel-hydrogen battery, AA size)
Interface	USB: USB1.1 compliant. Cable length: 1.5 m or shorter. Micro-USB B type terminal can connect to dedicated peripheral (using standard driver software).
Operating condition	Temperature: +10 to +40 °C Humidity: 30 to 85 %RH (no condensation)
Transport and storage conditions	Temperature: -20 to +60 °C Humidity: 10 to 95 %RH (no condensation)
Atmospheric pressure both for operation and storage condition	700 to 1060 hPa
Type of protection against electric shock	Internally powered equipment
Degree of protection against electric shock 	Type BF: The recorder, cuff and tubing are designed to provide special protection against electrical shock.
Dimensions	Approx. 95 (L) × 66 (W) × 24.5 (H) mm

Items	Descriptions
Mass	Approx. 120 g (excluding batteries)
Useful life	Recorder: 5 years. Self-authentication with internal data. Proper operation and maintenance in the best conditions. Durability varies with usage conditions.
Ingress protection	Device: IP22
Default mode	Continuous measurement
Restart time after defibrillation	Immediately
EMD	IEC 60601-1-2: 2014

Note:


- # Specifications are subject to change for improvement without prior notice.
- # Clinical trial for this device is performed in based on ISO 81060-2:2013.
- # The recorder is not medical device for monitoring patient. We don't recommend the way of use that has to monitor patient in real time at place like intensive care unit.

4.2. Dimensions

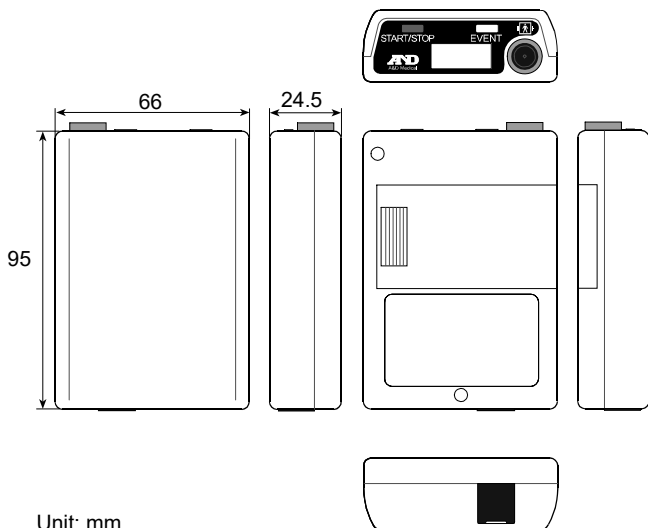

5. Component Names

5.1. Recorder

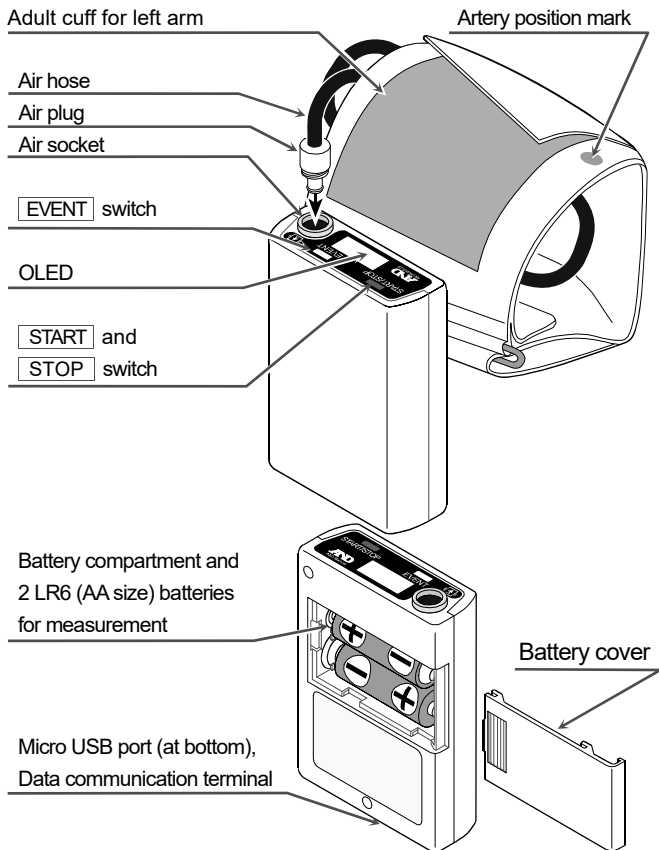

5.2. OLED Display (Organic light emitting diode)

Note


To get accurate diagnosis, take care to accurately read the data displayed on the recorder and interpret it properly.

The state of A-BPM is indicated on the OLED.

Clock time.

The state of settings and operation.

The measurement value of A-BPM.

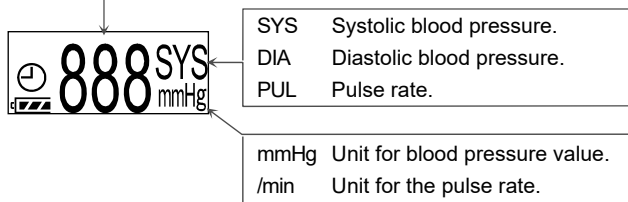

Refer to "3. Abbreviations & Symbols" for the meanings of symbols on the OLED.

Symbols	Meaning
	The mark is displayed during configuration.
	Displaying: A-BPM is performing.
	Memory full
	A-BPM sleep mark
	Battery indicator


5.3. Principal Switch Operations


5.3.1. A-BPM Operations

To start or suspend A-BPM.

Step 1. Store the preset program (of start times and intervals) for A-BPM.

Step 2. Press and hold the **EVENT** switch to switch between the following states.

"ON" A-BPM is started and the  mark is shown. Blood pressure measurements are performed in accordance with the preset A-BPM program.

"OFF" A-BPM is suspended and the  mark turns off. Manual blood pressure measurement can be performed by pressing the **START** switch.

To expand A-BPM interval time.

Step 1. Set the sleep mode to **"ON"** before the measurement.

Step 2. Start A-BPM by pressing and holding the **EVENT** switch.

The  mark is shown.


Step 3. When the **EVENT** switch is pressed during A-BPM, the interval time is doubled.

When the **EVENT** switch is pressed again, the interval time returns to basic value.

To stop during A-BPM

When the **START/STOP** switch is pressed during blood pressure measurement, the air is expelled immediately and the current measurement is stopped. However, A-BPM is continued. The next blood pressure measurement is performed in accordance with A-BPM settings.

To set the program for A-BPM.

- Step 1. If the display is hidden, press the **START/STOP** or **EVENT** switch to return to the waiting mode display.
- Step 2. If the  mark is shown, press and hold the **EVENT** switch to suspend A-BPM.
- Step 3. While pressing and holding the **START/STOP** switch, press and hold the **EVENT** switch until **Sleep** is displayed on the OLED.
- Step 4. The operation switches are as follows:
Refer to "8.3.1. A-BPM Items and Parameters"
EVENT switchChange the current parameter.
START/STOP switchDecision, next item, end of settings.

To measure blood pressure during A-BPM immediately. (Manual blood pressure measurement of A-BPM)

- Step 1. If the OLED is hidden, press the **START/STOP** or **EVENT** switch to return to the A-BPM waiting mode display. The A-BPM **waiting mode** is a state when blood pressure is not measured during the **interval time**.
- Step 2. Press the **START/STOP** switch during the A-BPM waiting mode.

To adjust the clock.

To set the monitor function of A-BPM.

Step 1. If the display is hidden, press the **START/STOP** or **EVENT** switch to return to waiting mode display.

Step 2. If the ⌚ mark is shown, press and hold the **EVENT** switch to suspend A-BPM.

Step 3. While pressing and holding the **START/STOP** switch, press and hold the **EVENT** switch until **Display** (after **Sleep**) is displayed on the OLED.

Step 4. Operation switches are as follows:

Refer to "8.2.2. The Clock and the Monitor Function of Measurement"

EVENT switchChange the current parameter.

START/STOP switchDecision, next item, end of settings.

5.3.2. Other Operations

To return from the waiting mode and show the monitor.

If the OLED display is hidden, press the **START/STOP** or **EVENT** switch to return to the waiting mode display.

Deleting measurement data

Step 1. If the display is hidden, press the **START/STOP** or **EVENT** switch to return to the waiting mode display.

Step 2. If the ⌚ mark is shown, press and hold the **EVENT** switch to suspend A-BPM.

Step 3. While pressing and holding the **START/STOP** switch, press and hold the **EVENT** switch until **DataClear** (after **Sleep** and **Display**) is displayed on the OLED.

Step 4. Select an operation.

- If you wish to delete data, press and hold the **START/STOP** switch.

Erasing blinks under **DataClear** on the OLED and the deletion of data starts.

Proceed to step 5 after deletion.

- If you keep (do not delete) data, press the **EVENT** switch and proceed to step 5.

Step 5. The recorder returns to waiting mode.


6. Blood Pressure Measurement Functions

The recorder is equipped with automatic blood pressure measurement (A-BPM) and can store measurement states and measurement results.

6.1. Automatic Blood Pressure Measurement (A-BPM)


Caution





When the A-BPM function is not used, suspend the function by pressing and holding the **EVENT** switch, so that the  mark turns off. Otherwise, the measurement will start at the next start time and the cuff may burst.

The A-BPM function measures the blood pressure at preset intervals using the built-in clock and stores the measurement result in the memory.

A-BPM can be started and suspended by pressing and holding the **EVENT** switch.

The  mark is displayed on the OLED while A-BPM is used. Blood pressure is measured automatically at the A-BPM start time.

When the  mark is displayed on the OLED, the initial pressurization value is set to AUTO, so that a proper pressurization value is selected automatically.

When the  mark is hidden, the initial pressurization value is set to 180 mmHg.

If the first pressurization is insufficient, re-pressurizations are performed automatically up to two times.

When you delete data from the memory or suspend A-BPM, the pressurization value is reset to the initial pressurization value.

When a measurement error occurs and the waiting time until the next start time is longer than 8 minutes, blood pressure is measured once after 120 seconds. The measurement result is stored in the memory.


If you want to suspend A-BPM, press and hold the EVENT switch.

6.1.1. A-BPM Waiting Mode

In the A-BPM waiting mode, the OLED shows the current time together with the ⌚ mark as follows.

In the waiting mode, the indicators are hidden automatically. Press any switch to show the display items.

The A-BPM **waiting mode** is a state when blood pressure is not measured during the interval time.

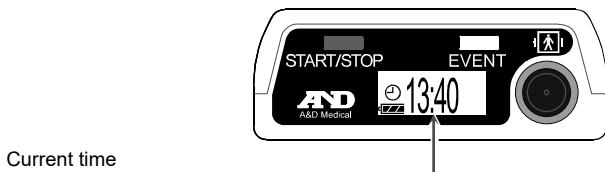


6.1.2. Sleep Function and Interval time

Set the sleep mode to "ON" in the preset program.

When the switch is pressed during A-BPM, the interval time doubles.

When the switch is pressed again in A-BPM, the interval time returns to its original length.

Refer to "8.3. A-BPM Preset Programs" for information on how to set the sleep mode.

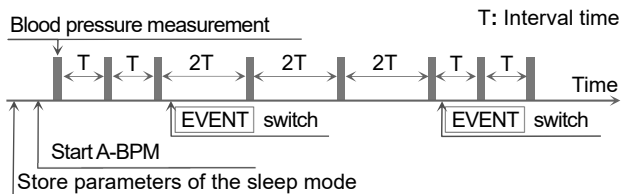

6.1.3. Stopping Measurement

When the switch is pressed during the blood pressure measurement, the air is expelled immediately and the current measurement is stopped. However, A-BPM is continued. The next blood pressure measurement is performed in accordance with A-BPM settings.

Note

- When measurement is stopped, the stop code is displayed on the OLED and is stored in the memory.

6.2. Measurement Results

6.2.1. Displaying Measurement Results

The monitor function can select "**Display ON**" or "**Display OFF**" for the A-BPM measurement result.

The content of the "**Display ON**" command includes "Pressure value during the measurement", "Measurement result" and "Error code for the measurement result".

When "**Display OFF**" is selected, the clock is displayed.

The factory setting is set to "**Display ON**".

Refer to "**8.2.2 The Clock and the Monitor Function of Measurement**".

6.2.2. Storing Measurement Results


Caution



Data processing of the measurement result

Do not use in a strong electromagnetic field.

The memory capacity for the measurement result is 600 data sets.

When the memory is full, the  mark is displayed and the recorder cannot perform measurement until data is deleted from the memory.

Note

Delete data from the memory before giving the recorder to a new patient. We recommend using the memory data of the recorder for each person separately. If the recorder memorizes data for multiple people, data may be difficult to process correctly.

6.2.3. Outputting Measurement Results

The measurement data stored in the memory can be output to the peripheral using USB data transfer.


Refer to "**8.7 Connecting the Recorder to a Dedicated Peripheral**".

Caution



Do not remove the cable while using USB communication. It may cause damage to the data.

Note

When the battery indicator displays , data transfer cannot be used. Replace the batteries to use data transfer.

6.2.4. ID numbers

The factory default ID number is "0".

Configure ID numbers using **dedicated peripheral**.



Note

ID numbers cannot be configured with the recorder and require use of a **dedicated peripheral**.


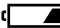
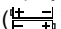
7. Preparing the Recorder

7.1. Installing the Batteries (Replacing Batteries)


Caution

- 
 - Install two new batteries with the correct "+" and "-" direction inside the battery compartment before attaching the recorder.
 - Replace both batteries at the same time.
 - Remove batteries from the recorder if it will not be used for a long period of time. Batteries may leak and cause a malfunction.
 - Use two alkaline batteries: type LR6 or designated rechargeable AA Ni-MH batteries.
 - When installing the battery in the battery compartment, first, push the spring terminal using the "-" terminal of the battery. Next, insert the "+" terminal.
If the battery is installed from the "+" terminal, the coating of the battery may be damaged by the spring terminal.
-  Do not mix different kinds of batteries or used batteries and new batteries. It may cause a leak, heating or damage.

Note


- When the level 1  of the battery level is displayed, replace with two new batteries before attaching the recorder.
- The recorder cannot perform blood pressure measurement or data transfer while the level 1  is displayed.
- When the battery and built-in battery are flat, nothing is displayed.
- Install batteries in accordance with the direction symbol ().

Procedure

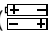- Step 1. Open the battery cover.
- Step 2. Remove the used batteries.
- Step 3. Refer to the direction symbol () inside of the battery compartment. Insert two new batteries in the proper "+" and "-" direction.

Push the spring terminal using the "-" terminal of the battery.

- Step 4. Insert the battery by pushing the "+" terminal.
- Step 5. Insert the second battery using the same method.
- Step 6. Close the battery cover.

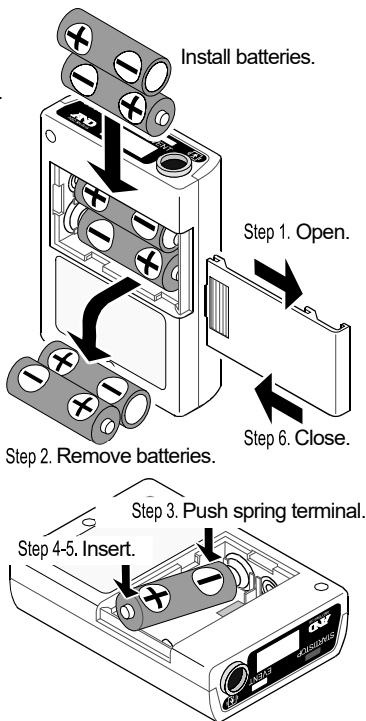

Caution



- Keep the batteries and battery cover away from infants and children to prevent accidental swallowing or other accidents.
- Use standard AA batteries. Do not use a swollen battery, rechargeable battery, or one that is wrapped in tape. It may become difficult to open the cover.

7.1.1. How to Replace Batteries


Measurement results and setting parameters are saved when the batteries are removed. When the built-in battery runs out charge, the date is reset to 01/01/2017 00:00.

Check and adjust the current time when the batteries are replaced. Refer to "**8.2.2. The Clock and the Monitor Function for Measurement**" to adjust the clock.

7.2. Preparing the Carry Holder

Note

When the carry holder is attached, use the accessory belt.
We recommend to use a belt to fit the recorder to the patient.

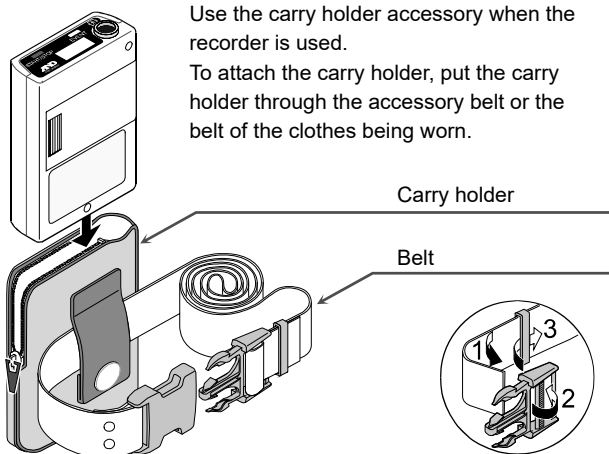

7.3. Inspection for Use

Caution



Inspect the recorder to maintain its performance, safety and effectiveness before use.

Confirm the following checklist before/after installing the batteries.

If a problem is found, stop using the recorder and attach a "**Malfunction**" or "**Do not use**" message. Contact your local dealer to repair it.

7.3.1. Battery Pre-installation Checklists

No.	Item	Description
1	Exterior	No damage and deformation due from dropping.
		No damage and looseness of switches, etc.
2	Battery	Check that the batteries are not flat. Replace with two new batteries before the patient uses it.
3	Cuff	Check that the cuff has not frayed. If the cuff is frayed, it may burst due to internal pressure.
4	Cuff connection	Check that there are no kinks and folding in the air hose.
		Check that the air socket and connector is connected firmly.
5	Attachments	Check that there is no damage to accessories. (Carry holder, belt, etc.)

7.3.2. Battery Post-installation Checklists

No.	Portion	Description
1	Battery	Check that there is no fire, smoke and strong smells.
		Check that there is no strange sound.
2	Display	Check that there is no strange display.
3	Operation	Check that the recorder operates correctly.
4	Measurement	Check that the measurement can be performed correctly and that the attachment of the cuff, the measurement, display and results are correct.


8. Operation

8.1. Operation Flowchart

Note

The initial settings (of the built-in clock, monitor function and initial pressurization value) and preset program for A-BPM do not need to be performed every time. Perform the settings when the recorder is used for the first time, when the settings have been lost, or when the settings should be changed.

These settings can be performed using a **dedicated peripheral**, too. Refer to the instruction manual for the ABPM Data Manager for details.

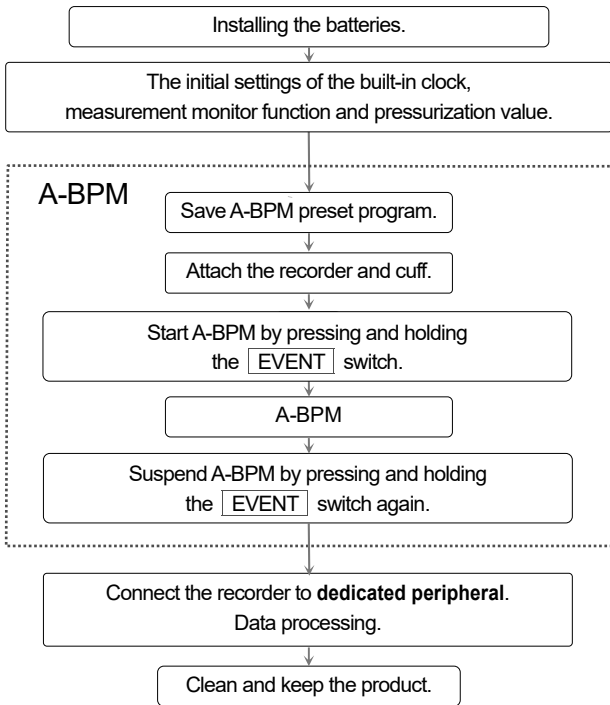

Complete procedure for use

8.2. Initial Settings

8.2.1. Factory Settings

The factory settings (initial settings) are described below:

Common items of the settings

Item	Factory setting
Monitor function	ON (indicated)
Year, Month, Day, Hour, Minute	Date of shipment

Items of A-BPM

Item	Factory setting
Sleep mode	OFF
Interval time when the sleep mode is ON	30 minutes
Start time for section 1	0 hours
Interval time for section 1	30 minutes
Start time for section 2	0 hours #1
Start time for the automated measurement	OFF
Operation time of the automated measurement	OFF

The content of the factory settings

When the EVENT switch is pressed and held, A-BPM starts. Blood pressure is measured every 30 minutes until A-BPM is suspended by pressing and holding the EVENT switch again.

- #1 : The settings between the interval time for section 2 and the interval time for section 6 are omitted because the start time for sections 1 and 2 is the same value.

8.2.2. The Clock and the Monitor Function of Measurement

The initial settings can be configured using the following methods.

- Using switches on the recorder.
- Using a **dedicated peripheral** that is connected to the recorder using the USB cable.

Procedure of operation using switches

Step 1. If the display is hidden, press the **START/STOP** or **EVENT** switch to return to the waiting mode display.

Step 2. If the ⌚ mark is shown, press and hold the **EVENT** switch to suspend A-BPM. The ⌚ mark turns off.

Step 3. While pressing and holding the **START/STOP** switch, press and hold the **EVENT** switch until **Display** (after **Sleep**) is displayed on the OLED.

Step 4. The operation switches are as follows:

EVENT switch Change the current parameter.


START/STOP switch Decision, next item, end of settings.
Then use these switches in another items.


Step 5. After configuring settings, press the **START/STOP** switch to return to waiting mode.

Item	OLED	Range
Monitor function	Display xx	xx = OFF, ON
Year	Clock Year xx	xx = 17 to 99. Last two digits of year.
Month	Clock Mon. xx	xx = 1 to 12 months
Day	Clock Day xx	xx = 1 to 31 days
Hour	Clock Hour xx	xx = 0 to 23 hours
Minute	Clock Min. xx	xx = 0 to 59 minutes

Enclosed characters : Factory settings and initial settings when batteries are consumed completely.

8.2.3. Initial Pressurization Value

When the  mark is displayed on the OLED, the initial pressurization value is set to AUTO, so that a proper pressurization value is selected automatically.

When the  mark is hidden, the initial pressurization value is set to 180 mmHg.


8.3. A-BPM Preset Programs

The initial settings can be configured using the following methods.

- Using the switches on the recorder.
- Using a **dedicated peripheral** that is connected to the recorder using the USB cable.



A-BPM can only be used while the automated measurement can be performed.

Note

Specify the **start time** and **interval** calculated from the time that the  mark is initially displayed on the OLED.

It is necessary to specify them again when using another A-BPM.

Operation using switches

- Step 1. If the display is hidden, press the **START/STOP** or **EVENT** switch to return to the waiting mode display.
- Step 2. If the  mark is shown, press and hold the **EVENT** switch to suspend A-BPM. The  mark turns off.
- Step 3. While pressing and holding the **START/STOP** switch, press and hold the **EVENT** switch until **Sleep** is displayed on the OLED.
- Step 4. Specify the sleep mode using the following switches.
If the sleep mode is "ON", proceed to step 5.
EVENT switchChange the current parameter.
START/STOP switchDecision, next item.
- Step 5. Specify the **start time** and **interval** in up to six sections using the following switches.
EVENT switchChange the current parameter.
START/STOP switchDecision, next item.
- Step 6. Specify the **start time** and **operation time** of the automated measurement using the following switches.
EVENT switchChange the current parameter.
START/STOP switchDecision, next item, end of settings.
- Step 7. After completing the settings, the recorder returns to its waiting mode.


Caution



Do not remove batteries while charging the settings.
If batteries are removed, input settings again.

8.3.1. A-BPM Items and Parameters

The preset program for A-BPM is as follows:

Item		OLED	Parameter	
Sleep mode		Sleep xx	xx = ON, <input type="checkbox"/> OFF	#1, #2
	Interval time	Cycle xx	xx = OFF, 5, 10, 15, 20, <input type="checkbox"/> 30, 60, 120 minutes	
Section 1	Start time	Hour 1 xx	xx = <input type="checkbox"/> 0 to 23 hours	
	Interval time	Cycle 1 xx	xx = OFF, 5, 10, 15, 20, <input type="checkbox"/> 30, 60, 120 minutes	
Section 2	Start time	Hour 2 xx	xx = <input type="checkbox"/> 0 to 23 hours	
	Interval time	Cycle 2 xx	xx = <input type="checkbox"/> OFF, 5, 10, 15, 20, 30, 60, 120 minutes	
Section 3	Start time	Hour 3 xx	xx = <input type="checkbox"/> 0 to 23 hours	
	Interval time	Cycle 3 xx	xx = <input type="checkbox"/> OFF, 5, 10, 15, 20, 30, 60, 120 minutes	
Section 4	Start time	Hour 4 xx	xx = <input type="checkbox"/> 0 to 23 hours	
	Interval time	Cycle 4 xx	xx = <input type="checkbox"/> OFF, 5, 10, 15, 20, 30, 60, 120 minutes	
Section 5	Start time	Hour 5 xx	xx = <input type="checkbox"/> 0 to 23 hours	
	Interval time	Cycle 5 xx	xx = <input type="checkbox"/> OFF, 5, 10, 15, 20, 30, 60, 120 minutes	
Section 6	Start time	Hour 6 xx	xx = <input type="checkbox"/> 0 to 23 hours	
	Interval time	Cycle 6 xx	xx = <input type="checkbox"/> OFF, 5, 10, 15, 20, 30, 60, 120 minutes	
	Start time	START xx	xx = <input type="checkbox"/> OFF, 0 to 23 hours	#3, #4
	Operation time	Operation xx	xx = <input type="checkbox"/> OFF, 1 to 27 hours	#3, #4

Automated measurement

Enclosed characters : Factory settings.

- #1 : When the sleep mode is set to "ON", the **start time** and **operation time** of the automated measurement and the **interval time** of the sleep mode can be used. The **interval time** for these sections (1 to 6) cannot be used.
- #2 : When the sleep mode is set to "OFF", the **interval time** for the sleep mode is not displayed.
- #3 : If the **start time** is specified and the **operation time** is set to "OFF", when the EVENT switch is pressed and held, the **automated measurement** starts at the preset **start time** and continues until the EVENT switch is pressed and held. If the EVENT switch is pressed and held again, the **automated measurement** continues immediately.

Note

When the **operation time** is specified, even if the EVENT switch is operated during the **automated measurement**, the **automated measurement** continues for the **operation time** from the time that the EVENT switch is initially operated.

- #4 : If the **start time** is set to "OFF" and the **operation time** is specified, when the EVENT switch is pressed and held, the **automated measurement** performs the first blood pressure measurement and continues for the **operation time**. If the EVENT switch is pressed and held during the **automated measurement**, it stops. If the EVENT switch is pressed and held again, the **automated measurement** is performed for the **operation time**.

Note

When the **start time** is specified and the **EVENT** switch is pressed and held during the **automated measurement**, it stops. When the **EVENT** switch is pressed and held again, the **automated measurement** is started immediately.

The content of the item

Sleep mode:

The **interval time** for the automated measurement can be specified. The **interval time** of sections 1 to 6 cannot be used. Refer to "**6.1.2 Sleep Function and Interval time**".

Section:

24 hours can be divided into six sections at most. Each section can specify the **start time** and **interval**. A-BPM can use only while the automated measurement can be performed.

Automated measurement:

All of the A-BPM can be controlled. Specify the **start time** and **operation time**. Refer to "**8.3.2. A-BPM Program Examples**".

8.3.2. A-BPM Program Examples

Example Start times and intervals. Simplified input.

Double sections


The **start time** for section 1 = 7:00

The **interval time** for section 1 = 15

The **start time** for section 2 = 22:00

The **interval time** for section 2 = 60

The **start time** for section 3 = 7:00 The same as section 1

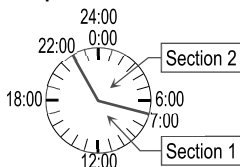

Section 3 and the following items are not displayed because the start time of section 3 is the same as for section 1.


When the **start time** for sections 2, 3, 4, 5 or 6 is the same as for section 1, these **start times** and **intervals** are not displayed.

Example 1 Automatic measurement

The **start time** for the automated measurement = OFF,

The **operation time** for the automated measurement = OFF.

After A-BPM is started, blood pressure measurement is performed according to the **start time** and **interval** of each section until A-BPM is suspended.

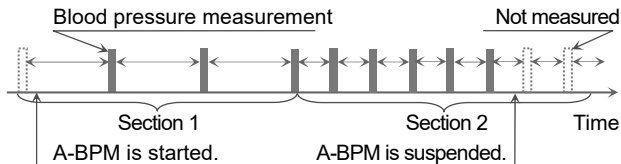


Example 2 Automatic measurement

The **start time** for the automated measurement = 7:00,

The **operation time** for the automated measurement = OFF.

After A-BPM is started, blood pressure measurement is started at 7:00. A-BPM is continued according to the **start time** and **interval** of each section until it is suspended.

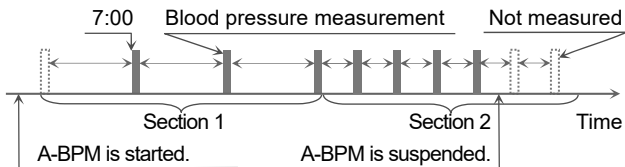


- # Even if the ⌚ mark is hidden once and is shown again during the **automated measurement**, the **automated measurement** continues.

Example 3 Automatic measurement

The **start time** for the automated measurement = OFF,

The **operation time** for the automated measurement = 26 hours.

After A-BPM is started, blood pressure measurement is performed according to the **start time** and **interval** for each section for 26 hours.

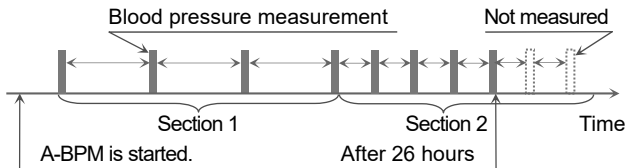

- # Even if the ⌚ mark is hidden once and is shown again during the **automated measurement**, the **automated measurement** does not continue beyond the **operation time**.

8.4. Deleting Measurement Data

Purpose of operation and explanation of function

Measurement data is deleted but the settings are not deleted.

The initial settings can be configured using the following methods.



- Using switches on the recorder.
- Using a **dedicated peripheral** that is connected to the recorder using the USB cable.

Caution



- If measurement data is deleted, it cannot be used again. Backup the data before deletion.
- Delete the measurement data for the last patient before the next patient uses the recorder.
- Deleting the data may take around ten seconds. Do not operate the device while the data is being deleted to ensure that it is deleted correctly.

Procedure of operation using switches

- Step 1. If the display is hidden, press the **START/STOP** or **EVENT** switch to return to the waiting mode display.
- Step 2. If the  mark is shown, press and hold **EVENT** to suspend A-BPM. The  mark turns off.
- Step 3. While pressing and holding the **START/STOP** switch, press and hold the **EVENT** switch until **DataClear** (after **Sleep** and **Display**) is displayed on the OLED.
- Step 4. Select an operation.
- If you wish to delete data, press and hold the **START/STOP** switch. **Erasing** blinks under **DataClear** on the OLED and the deletion of data starts. Proceed to step 5 after deletion.
 - If you keep (do not delete) data, press the **EVENT** switch and proceed to step 5.
- Step 5. The recorder returns to the waiting mode.

8.5. Attaching the Product to the Patient

8.5.1. Information for Patients


Explain the following to the patient so that they can use the recorder safely.

Precautions during the blood pressure measurement

- Relax the arm and stay quiet when inflation begins.
- Stay in the same position throughout measurement.
- Avoid vibration and noise during measurement.
- Blood pressure is measured for approximately 1 minute after pressurization. Stay still until measurement finishes.
The measurement process between inflating the cuff to releasing the air requires up to 170 seconds.
- The recorder may re-inflate to measure the blood pressure again after the end of pressurization. This may be caused by body motion, etc.
- The recorder may start blood pressure measurement after approximately 120 seconds when measurement data is invalid and the next measurement is 8 minutes later. This may be caused by body motion, etc.
- The recorder may obstruct vehicle and machine operation.
Avoid vehicle and machine operation while wearing the recorder.

How to stop or suspend the measurement


Press the **START/STOP** switch to stop blood pressure measurement. An error code is stored in the memory. Blood pressure is measured again after 120 seconds. For A-BPM, only the current blood pressure measurement is stopped, and measurement will be performed at the next **start time**.


To suspend A-BPM, press and hold the **EVENT** switch, so that the  mark turns off.

Remove the cuff if the current blood pressure measurement cannot be stopped using the **START/STOP** switch.

Caution



- Press the **START/STOP** switch to stop the blood pressure measurement. An error code is stored in the memory.
During A-BPM, only the current blood pressure measurement is stopped and measurement will be performed at the next **start time**.
- When pain in the arm or an unexpected condition occur, stop the measurement, remove the cuff and consult a doctor.
Suspend A-BPM by pressing and holding the **EVENT** switch, so that the  mark turns off.

Press and hold the **EVENT** switch again to resume A-BPM automated measurement. The  mark is shown on the OLED. Data is recorded continuously, except during the suspended period.

How to use manual measurement during A-BPM

The procedure for temporary measurement that is not included in preset program.


- Step 1. If the OLED display is hidden, press the **START/STOP** or **EVENT** switch to return to the A-BPM waiting mode display.
- Step 2. Press the **START/STOP** switch to immediately measure the blood pressure during A-BPM.
- Step 3. Measurement results are stored in the memory.

When the **START/STOP** switch is pressed during measurement, the measurement is suspended.

Precautions when wearing the recorder

- The recorder is a precision instrument. Do not drop or jolt the recorder.
- The recorder and cuff are not waterproof (water resistant). Protect the product from contact with rain, sweat and water.
- Do not put anything on the product.
- When the cuff is moved by excessive motion and exercise, attach the cuff again.
- Arrange the air hose, so that kinks do not form and so that it does not wrap around your neck at bedtime.

Replacing Batteries

When the  mark is displayed, the recorder cannot measure blood pressure or communicate with a **dedicated peripheral**. Replace with two new batteries immediately.

8.5.2. Cuff Cover


Note

Keep the cuff and cuff cover clean.

- Change the cuff cover for each person.
- Use the appropriate optional cuff covers.

8.5.3. Attaching the Cuff, Carry holder and Recorder

Caution

- Do not attach the cuff if the patient has dermatitis, external wounds, etc.
- Remove the cuff and cease use if dermatitis or other symptoms appear on the patient.
-  Prevent the air hose from coiling around the neck and body.
- Take care when using around infants, as there is a danger of suffocation.
- Insert the connector of the air hose firmly until the end of its rotation. If the connection is incorrect, it may cause air leakage and a measurement error.

Note


- Attach the cuff in the correct position and wrap around the arm to measure blood pressure correctly.
- Prevent the cuff and air hose from vibrating during measurement. The recorder measures a delicate change in the air pressure inside the cuff.
- The accessory cuff is an adult cuff for the left arm. If the cuff size does not fit, purchase an optional cuff.


	Arm circumference	
Small cuff	15 to 22 cm	5.9" to 8.7"
Adult cuff	20 to 31 cm	7.8" to 12.2"
Large cuff	28 to 38 cm	11.0" to 15.0"
Extra large cuff	36 to 50 cm	14.2" to 19.7"

- Keep the cuff clean.
- We recommend that the patient uses the carry holder and belt.
- The cuff is not made from natural rubber latex.


How to put on the cuff, recorder and holder

- Step 1. Pass the end of the cuff through the ring and make a bracelet shape.
- Step 2. Find the brachial artery in the left arm using palpation.
- Step 3. Attach the cuff directly against the skin, so that the white mark is directly over the brachial artery and the lower edge of the cuff is put on approximately 1 - 2 cm above the inside of the elbow.
- Step 4. Wrap the cuff, so that the ring is within the range, is flat and does not slip down, but there is room to insert two fingers.
- Step 5. Fix the air hose using adhesive tape, so that it passes over the shoulder.
- Step 6. Pass the belt through the carry holder.
- Step 7. Adjust the belt so that the carry holder is on the left side.
- Step 8. Connect the air plug to the air socket on the recorder.
- Step 9. Put the recorder into the carry holder.

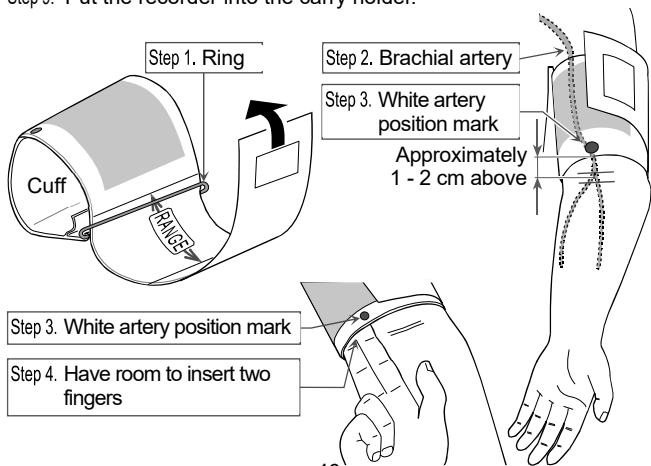


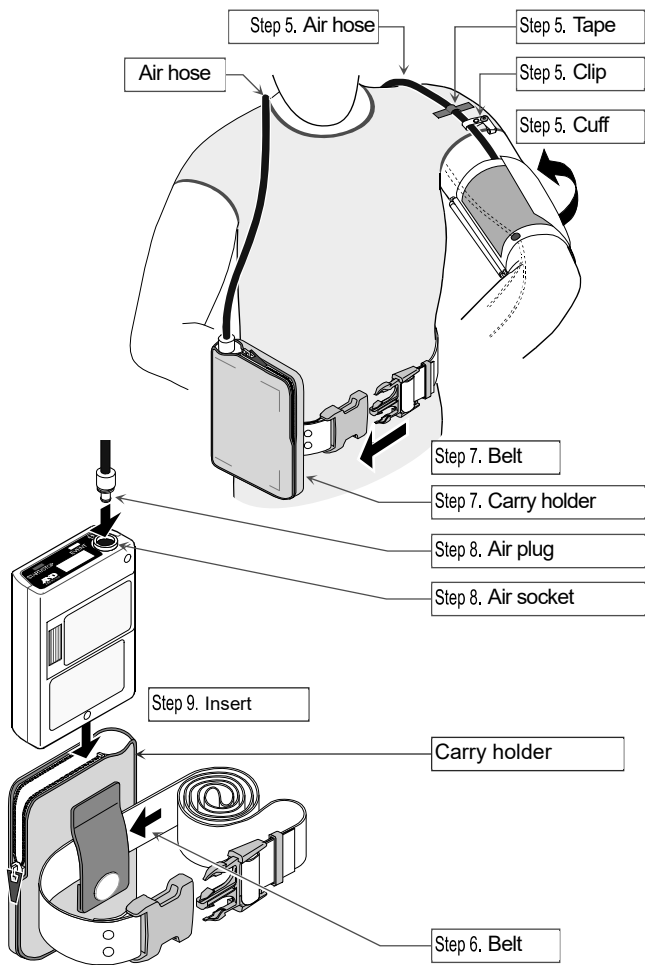


Cuff

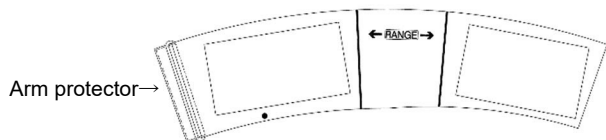

Correct

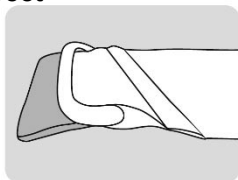


Arm protector flat against the arm.

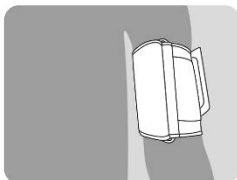


Arm is protected

Incorrect

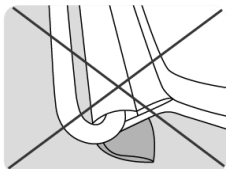

Arm protector folded under the arm.

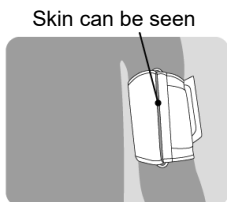

Skin can be seen


Arm is not protected

8.6. Blood Pressure Measurement Operations

8.6.1. A-BPM Operations


When A-BPM is started, the blood pressure is measured in accordance with the preset parameters.

Note

- Set the built-in clock and initial pressurization value before measurement, as A-BPM uses them. Refer to **"8.2.2. The Clock and the Monitor Function of Measurement"** and **"8.3. A-BPM Preset Programs"**.
- When the recorder is removed, suspend A-BPM by pressing and holding the **EVENT** switch.
If the recorder is removed during A-BPM, the inflation of the cuff starts at the next **start time** and the cuff may burst.
When A-BPM is resumed, press and hold the **EVENT** switch again.
- The  mark is displayed while A-BPM is used.
- Manual blood pressure measurement can be performed during A-BPM waiting mode.
- The measurement result for the manual blood pressure measurement can be stored in the memory.
- When A-BPM is stopped, the error code **E07** is displayed on the OLED and stored in the memory.


To start A-BPM

Step 1. Press and hold the **EVENT** switch.

Step 2. The  mark is shown on the OLED. A-BPM is started.

To suspend A-BPM

Step 1. Press and hold the switch.

Step 2. The  mark is hidden. A-BPM is suspended.

To Stop during A-BPM

When the switch is pressed during blood pressure measurement, the air is expelled immediately and the current measurement is stopped. However, A-BPM is continued. The next blood pressure measurement is performed in accordance with A-BPM settings.

To measure blood pressure during A-BPM immediately (Manual blood pressure measurement of A-BPM)

Step 1. If the OLED display is hidden, press the or switch to return to the A-BPM waiting mode display. The A-BPM **waiting mode** is a state whereby blood pressure is not measured during the **interval time**.

Step 2. Press the switch during the A-BPM waiting mode.

To double the interval time or reset it

When sleep mode is "ON" and the switch is pressed during A-BPM waiting mode, the interval time is doubled.

8.6.2. Manual Measurement

Use manual blood pressure measurement for a tentative test measurement and immediate blood pressure measurement.

Note

- Manual blood pressure measurement can start immediately in a waiting mode.
- The measurement result is stored in the memory.

To measure blood pressure during A-BPM immediately. (Manual blood pressure measurement for A-BPM)

- Step 1. If the OLED display is hidden, press the **START/STOP** or **EVENT** switch to return to the A-BPM waiting mode display. The A-BPM **waiting mode** is a state that blood pressure is not measured during the **interval time**.
- Step 2. Press the **START/STOP** switch during A-BPM waiting mode.

8.6.3. Stopping and Suspending Measurements


The A-BPM function can be suspended when necessary. Ongoing A-BPM or manual blood pressure measurement can be stopped immediately.

Note

When blood pressure measurement is stopped, the stop code **E07** is displayed on the OLED and is stored in the memory.

To suspend A-BPM

Step 1. Press and hold the **EVENT** switch.

Step 2. The  mark is hidden. A-BPM is suspended.

To stop ongoing blood pressure measurement

When the **START/STOP** switch is pressed during blood pressure measurement, the air is expelled immediately and the current measurement stops.

However, A-BPM continues. The next blood pressure measurement is performed in accordance with the A-BPM settings.


8.7. Connecting the Recorder to a Dedicated Peripheral

8.7.1. Connecting with a USB cable

Refer to the instruction manual of ABPM Data Manager concerning of the communication settings.


Caution

Connecting the cable

- Connect an authorized USB cable to the micro USB terminal.
- Insert the cable in the correct direction. Improper connection may cause failure and malfunction. Confirm that the terminal cable is properly connected.
- Blood pressure cannot be measured during USB communication.
-  Do not attach to the patient when the recorder is connected to the cable. The cable may become wrapped around the body or neck.


Preparation of a dedicated peripheral

Remove the recorder and cuff from the patient before connecting the recorder (TM-2440) to a **dedicated peripheral**.

- If the level 1  is displayed, connect the recorder (TM-2440) to the peripherals after replacing the batteries.

To connect the recorder to a dedicated peripheral using the USB cable

- Step 1. Open the micro USB terminal on the recorder.
Connect the accessory USB cable.

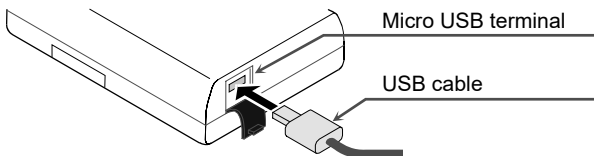

To start data communication with a dedicated peripheral

- Step 1. Connect the micro USB cable between the recorder and the **dedicated peripheral**.
- Step 2. The buzzer will sound and the following symbol appears on the OLED. The data communication enters standby mode.



- Step 3. Carry out analysis using the **dedicated peripheral**.
The data communication only enters active online mode during USB communication.



To stop data communication with a dedicated peripheral

- Step 1. Remove the cable in the standby mode.

9. Maintenance

9.1. Product Storage, Inspection and Safety Management

Medical instruments, such as this recorder, must be managed so that they function properly when necessary and reliably ensure the safety of the patient and the operator. As a basic rule, the operator should inspect this instrument daily, such as by following the "Inspection before use".

Daily management, such as inspection before use, is necessary to maintain the performance, safety and effectiveness of the recorder.

We recommend a periodic inspection of the recorder every year.

Note
Medical institutions must perform maintenance management to ensure the safe use of the medical instrument.

9.2. Cleaning the Product

Caution



- ❑ Clean the recorder before use and after use.
Clean the recorder before attaching to the next patient.
- ❑ Do not spray water and immerse in water to clean the recorder.
This may cause a malfunction.
- ❑ Dry the recorder after wiping it with water and the antiseptic solution, so that the liquid does not penetrate the recorder.
- ❑ Disinfect the recorder periodically to ensure infection prevention.
Do not use a sterilizer on the recorder.
- ❑ Do not use an organic solvent (for example: thinner) or povidone-iodine solution to clean the recorder. It may cause discoloration, damage and malfunction.
- ❑ Do not use a hair dryer etc. to dry the recorder. It may cause malfunction and damage.

Confirmation after Cleaning the Cuff

Confirm that the cuff bladder is correctly inserted inside the cuff cloth. If it is not correctly inserted, damage or explosion may occur during inflation.

Cleaning the recorder

Wipe the dirt and dust from the exterior case of the recorder using gauze moistened with water or warm water and squeezed well. When blood or medicines, etc. adhere to the case, firstly, clean with gauze moistened with antiseptic solution and squeezed well. Then, wipe the wet case using gauze moistened with water or warm water and squeezed well.

We recommend using chemicals (ingredient name) from the antiseptic solution listed in the table (**Example of useable antiseptic solution (Ingredient name)**).

Cleaning the cuff

When you clean and disinfect the cuff cover and cuff cloth, remove the cuff bladder inside the cuff cloth. Clean the dirt and dust using gauze moistened with water or warm water and squeezed well.

Refer to the antiseptic solutions in the table (**Example of useable antiseptic solution (ingredient name)**) when disinfecting.

Example useable antiseptic solution (Ingredient name)

Component Name	Product Name
Benzalkonium chloride	Benzalkonium chloride 0.1% solution
Isopropanol	70% in 1-propanol
Ethanol	Ethanol for disinfection 76.9 to 81.4 vol%
Chlorhexidine Gluconate	Chlorhexidine Gluconate 0.5% Solution

- Wipe lightly after squeezing the gauze soaked in the antiseptic solution.
- After that, wipe off the antiseptic solution with gauze that has been soaked in water or lukewarm water and squeezed well. And wipe off the moisture with a soft, dry cloth.
- After that, dry it thoroughly before use.
- For the dilution rate, follow the instructions on the product's precautionary statement and use it as an aqueous solution.

Note

The cuff and air hose are consumables.

If a measurement error occurs frequently or blood pressure cannot be measured, replace them with new ones.

Refer to "エラー! 参照元が見つかりません。 . **Optional Items (requiring order)**" in this manual.

9.3. Periodic Inspection

Perform the daily periodic inspection to use the recorder correctly. The inspection is described below:

9.3.1. Battery Pre-installation Inspection

Items	Description
Exterior	No damage or deformation from dropping.
	No dirt, rust and scratches on any part.
	No cracking or rattling of the panel.
Operation	No damage or rattling of switches and buttons.
Display	No dirt or scratches on the display panel.
Measurement Cuff	<input type="checkbox"/> The air hose must not be folded. If air remains in the cuff, it may cause peripheral dysfunction from stopping the blood flow in the arm.
	<input type="checkbox"/> The cuff bladder is correctly inserted inside the cuff cloth.
	<input type="checkbox"/> No fraying of the cuff. The cuff doesn't ravel.
	Replace the cuff when a problem is found.
	The cuff is disposable.
	<input type="checkbox"/> If there is a crack or adhesive matter in the connection between the cuff and cuff bladder.
	<input type="checkbox"/> If the air hose loses its flexibility and becomes hard.
<input type="checkbox"/> When the surface of the air hose becomes glossy or feels oily.	
<input type="checkbox"/> When the air bladder has cracks. We recommend replacing cuffs every three years, regardless of frequency of use.	
Wearing tools	No damage in the carry holder, belt and cuff.
Connection	The air plug is connected to the air socket correctly.

9.3.2. Battery Post-installation Inspection

Item	Description
Exterior	No fire, smoke or strong smells.
	No strange sounds.
Operation	No trouble with the functioning of switches and buttons.
Measurement Cuff	Measurement values are close to the usual value.
	No strange sounds or actions during measurement.
Inspection of blood pressure value	If blood pressure values are incorrect, contact your local dealer.

9.4. Disposal



Follow the laws of the local government for environment protection for the disposal and recycling of the product.

Disposal of the cuff

The cuff worn on the patient is medical waste.

Dispose of it properly as medical waste.

Disposal of the rechargeable built-in battery

 Caution	
	The recorder is equipped with a backup battery inside. When disposing of the recorder, dispose of the battery properly in accordance with the local regulations for environmental protection.

Others

Name	Part	Material
Package	Case	Cardboard
	Cushion	Air cushion, special case
	Bag	Vinyl
Inside the recorder	Case	ABS + PC resin
	Internal parts	General parts
	Chassis	Iron
	Backup battery on the board	Lithium rechargeable coin cell battery: ML2016H

	Battery	Alkaline battery: 1.5V LR6 or AA size Rechargeable battery: AA size Ni-MH batteries, 1900 mAh or more
--	---------	---

9.5. Troubleshooting

Consult the following checklist and error code list before contacting your local dealer.

If these measures do not rectify the problem or the problem occurs again, contact your local dealer.

Problem	Main cause	Treatment
No display after pressing any switches.	Battery power has been consumed.	Replace with new batteries.
No OLED during A-BPM.	OLED may disappear due to the electrostatic effect.	Remove batteries and reinstall them.
Frequent clock reset.	The backup battery does not charge. #1	Charge for 48 hours using new batteries.
No pressurization	Cuff is not connected correctly.	Check the cuff and air hose for folding, kinks and connection.
No USB communication #2	The communication cable is removed.	Confirm that the cable is connected correctly.
Battery cover cannot be opened	Non-standard size batteries were used.	Contact your local dealer.

#1: Users (unauthorized maintenance personnel) cannot replace the backup battery (lithium battery) on the circuit board inside the recorder. The backup battery is charged from the batteries (LR6 or AA size) for measurement.

#2: A **dedicated peripheral** is required.



Caution



Do not disassemble or modify the recorder. It may be damaged.

9.6. Error Codes

Measurement error codes

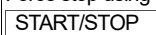Code	Meaning	Cause and treatment
E03	Pressure zero error	Release the air left in the cuff.
E04	Low battery	Replace with new batteries.
E05	Failure of pressurization	<ul style="list-style-type: none"><input type="checkbox"/> Inflation does not reach the target pressure.<input type="checkbox"/> Check the cuff connection.<input type="checkbox"/> If there are no problems with the cuff connection, the recorder may have malfunctioned and requires inspection.
E06	Pressure exceeds 299 mmHg	Body motion may occur during pressurization. Relax and keep still during measurement. If this does not help, inspect the recorder.
E07	Force stop using  switch.	Press the  switch only when necessary.
E08	Blood pressure cannot be measured.	<ul style="list-style-type: none"><input type="checkbox"/> The heartbeat cannot be detected due to body motion or noise from clothes.<input type="checkbox"/> Relax and do not move.<input type="checkbox"/> Confirm the position of the cuff.<input type="checkbox"/> If this failure occurs even when relaxed, contact your dealer to inspect and repair the recorder.

Code	Meaning	Cause and treatment
E 10	Excessive body motion.	Relax and keep still during measurement.
E20	Out of range, $30 \leq \text{PUL} \leq 200$	If these errors occur multiple times, try another blood pressure measurement. #1 PP = SYS - DIA SYS: Systolic blood pressure DIA: Diastolic blood pressure PP: Pulse pressure
E21	Out of range, $30 \leq \text{DIA} \leq 160$	
E22	Out of range, $60 \leq \text{SYS} \leq 280$	
E23	Out of range, $10 \leq \text{PP} \leq 150$ #1	
E30	Measurement is above 180 seconds.	If the inflation speed or exhaust speed is slow, an inspection is necessary.
E31	Exhaust is above 90 seconds.	The exhaust speed may be slow, an inspection is necessary.
E48	Heartbeat cannot be detected.	Heartbeat cannot be detected because of body motion, etc. Measure the blood pressure while relaxed and do not move.
E60	The settings of the interval time are incorrect.	If the interval time is set to 120 minutes, the difference between last start time and next start time cannot divide into two hours perfectly.
E90	Zero pressure error for safety circuit.	<input type="checkbox"/> Displays at the measurement start time. <input type="checkbox"/> Release the air remaining in the cuff completely.

Code	Meaning	Cause and treatment
E91	Safety circuit detects over load pressure.	<ul style="list-style-type: none"> ❑ Body motion may be detected during pressurization. Relax and do not move during the measurement. ❑ If this error occurs even when relaxed and not moving, contact your dealer for inspection.

Hardware error codes on the recorder

Code	Meaning	Cause and treatment
E52	Memory error	<ul style="list-style-type: none"> ❑ It may happen in case of a strong impact, such as dropping the recorder. ❑ If this code displays frequently, there is a malfunction in the built-in memory. Contact your dealer for inspection.

Note

The error codes may be changed without any notice.

10. Optional Items (requiring order)

Cuffs

Name	Description	Order code
Small cuff for left arm	Arm circumference 15 to 22 cm 5.9" to 8.7"	TM-CF202D
Adult cuff for left arm	Arm circumference 20 to 31 cm 7.8" to 12.2"	TM-CF302D
Large cuff for left arm	Arm circumference 28 to 38 cm 11.0" to 15.0"	TM-CF402D
Extra-large cuff for left arm	Arm circumference 36 to 50 cm 14.2" to 19.7"	TM-CF502D
Adult cuff for right arm	Arm circumference 20 to 31 cm 7.8" to 12.2"	TM-CF802D
Disposable cuff	10 sheets	TM-CF306A
Small cuff cover	for left arm 10 sheets	AX-133024667-S
Adult cuff cover	for left arm 10 sheets	AX-133024500-S
Large cuff cover	for left arm 10 sheets	AX-133024663-S
Extra-large cuff cover	for left arm 10 sheets	AX-133024503-S
Adult cuff cover	for right arm 10 sheets	AX-133024353-S
Small cuff cloth	for left arm 2 sheets	AX-133039854-S
Adult cuff cloth	for left arm 2 sheets	AX-133039658-S
Large cuff cloth	for left arm 2 sheets	AX-133039855-S
Extra-large cloth	for left arm 2 sheets	AX-133039856-S
Adult cuff cloth	for right arm 2 sheets	AX-133039857-S
Air hose adaptor	–	TM-CT200-110B

Data analysis

Name	Description	Order code
USB cable	–	AX-KOUSB4C


Others

Name	Description	Order code
Activity record sheet	10 sheets	AX-PP181-S
Carry holder	–	AX-133025995
Belt	–	AX-00U44189
Clips	5 pieces	AX-110B-20-S

11. Appendix

11.1. Principle of Blood Pressure Measurement


Measurement procedure: Wrap the cuff around the upper arm. Inflate the cuff to a pressure exceeding the systolic blood pressure. Then, exhaust the air from the cuff gradually. While the pressure is detected in the cuff in the air exhaustion stage, the pulse waveform appears in synchronization with the heartbeat. The pulse waveform suddenly increases near the systolic blood pressure. It increases further with exhaustion until it reaches its highest amplitude, then decreases gradually. The changes in the pulse waveform are illustrated on the next page. In oscillometric blood pressure measurement, the systolic blood pressure is specified as the point where the amplitude increases suddenly after the pulse in the cuff pressure is detected, while the mean blood pressure is specified as the point where the amplitude reaches its maximum and the diastolic blood pressure is specified as the point where the amplitude decreases gradually. Actually, the pressure sensor detects subtle changes in the cuff pressure over time, stores the pulse waveform in memory and evaluates the systolic and diastolic blood pressures according to the oscillometric measurement algorithm. The details in the algorithm vary with the blood pressure monitor. Blood pressure values for adults and infants are measured by the oscillometric method and are compared with those measured by the auscultatory method. Diastolic blood pressure is defined as the end point of phase 4 in the auscultatory method. The pulse waveform of the cuff pressure depends on the characteristics of the cuff material. Therefore, by using the specified cuff and the measurement algorithm, the measurement accuracy is maintained. The air hose length is less than 3.5 m, because of the damping characteristics due to pulse wave propagation.

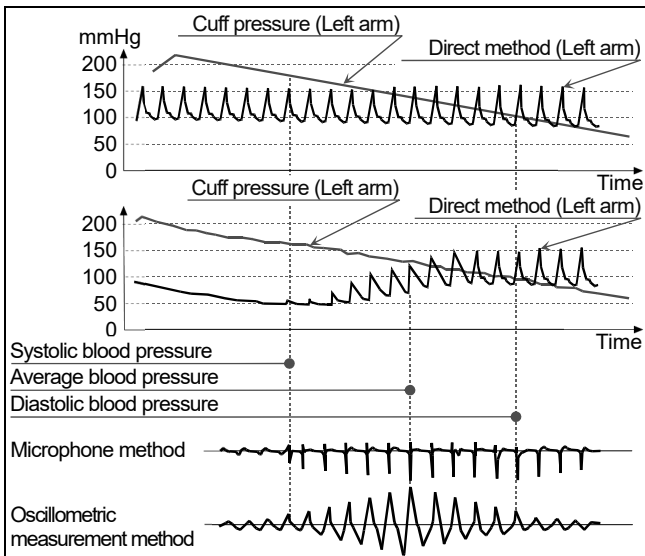

Blood pressure measurement error factors

The pulse graph can be an objective indicator of the reliability of the measurement accuracy.

When noise occurs due to irregular heart beat or physical movements, the amplitude of the graph changes. When the pulse graph is not a smooth outline, check again or use other methods.

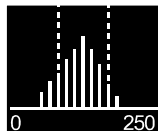

Pulse graph

Cuff position at the same height as heart

Wrap the cuff around the arm at the same level as the heart. If the cuff position is incorrect, a measurement error occurs. For example, if the cuff is 10 cm lower than the heart level, the blood pressure is measured 7 mmHg higher.

Proper cuff size

Use a cuff of adequate size. If it is too small or too big, a measurement error occurs. Measurements with a cuff that is too small tend to be evaluated as high blood pressure, regardless of the proper blood pressure and normal arteries. Measurements with too large a cuff tend to be evaluated as low blood pressure, especially for those who suffer from severe arteriosclerosis or have abnormal arterial valves. The wrong cuff size is a cause of differences between the direct method and oscillometric measurement method. The cuff has the range of the arm circumference shown on the label. Select and attach the proper size cuff for each patient. The accuracy of the blood pressure measurement is guaranteed by the pressure accuracy of the pressure sensor, exhaust characteristics and measurement algorithm, so long as the proper cuff and air hose are used. Inspect the pressure accuracy of the pressure sensor and exhaust characteristics periodically.

11.2. EMD Information



The requirements that apply to medical electronic instruments are described below:

Performance within the EMD guidelines

The use of the recorder require special precautions for EMD (Electromagnetic Disturbances). Operate the recorder in accordance with the warnings for EMD described in this manual. Portable and mobile RF communication equipment (e.g. cell phones) can affect medical electrical equipment.

Accessories compliant with EMD standards

The accessories and options for this recorder meet the conditions of IEC60601-1-2:2014. If an unauthorized accessory is used, it may cause increasing emissions and lower noise immunity.

 Warning	
	Use accessories designated by the A&D company. Unauthorized accessories may be influenced by electromagnetic emission and have reduced immunity against disturbances.

EMISSIONS LIMITS

Phenomenon	Compliance
Radiated RF emission CISPR11	Group 1, Class B

IMMUNITY TEST LEVELS: Enclosure Port

Phenomenon	Immunity test levels
Electrostatic discharge IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air
Radiated RF EM fields IEC 61000-4-3	10 V/m 80 MHz - 2.7 GHz 80 % AM at 1 kHz
Proximity fields from RF wireless communications equipment IEC 61000-4-3	See table (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)
Rated power frequency magnetic fields IEC 61000-4-8	30 A/m 50 Hz / 60 Hz

IMMUNITY TEST LEVELS: PATIENT COUPLING Port

Phenomenon	Immunity test levels
Electrostatic discharge IEC 61000-4-2	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air

IMMUNITY TEST LEVELS: Signal input/output Port

Phenomenon	Immunity test levels
Electrostatic discharge IEC 61000-4-2	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air
Electrical fast transients/bursts IEC 61000-4-4	± 1 kV 100 kHz repetition frequency
Conducted disturbances induced by RF fields IEC 61000-4-8	3 V 0.15 MHz - 80 MHz 6 V in ISM and amateur radio bands between 0.15 MHz and 80 MHz 80% AM at 1 kHz

Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment

Test frequency (MHz)	Band (MHz)	Service	Modulation	Maximum power (W)	Distance (m)	Immunity test level (V/m)
385	380 - 390	TETRA 400	Pulse modulation 18 Hz	1.8	0.3	27
450	430 - 470	GMRS 460 FRS 460	FM ± 5 kHz deviation 1 kHz sine	2	0.3	28
710	704 - 787	LTE Band 13,17	Pulse modulation 217 Hz	0.2	0.3	9
745						
780						
810	800 - 960	GSM 800/900 TETRA 800 iDEN 820 CDMA 850 LTE Band 5	Pulse modulation 18 Hz	2	0.3	28
870						
930						
1720	1700 - 1990	GSM 1800 CDMA 1900 GSM 1900 DECT LTE Band 1,3,4,25 UMTS	Pulse modulation 217 Hz	2	0.3	28
1845						
1970						
2450	2400 - 2570	Bluetooth WLAN 802.11 b/g/n RFID 2450 LTE Band 7	Pulse modulation 217 Hz	2	0.3	28
5240	5100 - 5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0.2	0.3	9
5500						
5785						

12. Limited Warranty

Monitor Warranty Term: 2 Years

Cuff & Accessory Warranty Term: 1 Year

Limited Warranty:

A&D Medical (“A&D”) warrants to the first purchaser (“the Owner”) that the A&D product the Owner purchased (the “Product”) will be free from defects in material, workmanship and design for the applicable Warranty Term stated above from the date the Owner purchased the Product under normal use. This Limited Warranty is personal to the Owner and is not transferable. If the Owner purchased the product from A&D directly and the Product is defective, then the Owner should return the Product to A&D in accordance with the procedure set forth below. If the Owner purchased the Product from a distributor or dealer, the Owner should contact the distributor or dealer for service.

A&D’s warranty obligation is limited to the repair or replacement, at A&D’s option, of the defective Product that has been returned by the Owner within the warranty period. Such repair or replacement will be at no charge to the Owner. The repaired or replacement Product is warranted here-under for the longer of the remainder of the original warranty period or 90 days from the date of shipment of the repaired or replacement Product. To obtain a warranty service, please contact us in the **US at 1-888-726-9966** or in **Canada at 1-800-461-0991** for return address, shipping and handling fee, and other instructions for processing the warranty. Please ensure you

have satisfactory proof of the date of Owner's purchase and a description of the defect.


Returns will not be accepted unless a Return Material Authorization (RMA) Number has been issued from A&D Customer Service Representative.

This Limited Warranty does not cover, and A&D will not be liable for (i) any shipment damage, (ii) any damage or defect due to misuse, abuse, failure to use reasonable care, failure to follow written instructions enclosed with the Product, accident, subjecting the Product to any voltage other than the specified voltage, improper environmental conditions, or modification, alteration or repair by anyone other than A&D or persons authorized by A&D, or (iii) expendable or consumable components.

THIS LIMITED WARRANTY IS THE ONLY WARRANTY PROVIDED BY A&D; THERE ARE NO OTHER EXPRESS WARRANTIES. If A&D cannot reasonably repair or replace the Product, A&D will refund the amount the Owner paid for the Product (not including taxes), less a reasonable charge for usage. To receive a refund, the Owner must have returned the Product and all associated materials to A&D. The above remedy of repair, replacement or refund is the Owner's only and exclusive remedy. IN NO EVENT SHALL A&D BE LIABLE FOR ANY DAMAGES, INCLUDING, WITHOUT LIMITATION, LOST PROFITS, LOST INFORMATION OR REPLACEMENT COSTS, ARISING OUT OF THE OWNER'S USE OF OR INABILITY TO USE THE PRODUCT, INCLUDING, WITHOUT LIMITATION, ANY SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES, EVEN IF A&D

HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. Some states do not allow the exclusion of incidental or consequential damages, so that the above exclusions may not apply to the Owner. This warranty gives the Owner specific legal rights, and the Owner may also have other rights that may vary from state to state. No distributor, dealer or other party is authorized to make any warranty on behalf of A&D or to modify this warranty, or to assume for A&D any liability with respect to its products.



 Manufactured by/ Fabricado por/ Fabriqué par:

A&D Company, LTD.

1-243 Asahi, Kitamoto-shi, Saitama-ken 364-8585 JAPAN

Telephone: [81] (48) 593-1111 Fax: [81] (48) 593-1119

Manufactured for/ Fabricado para/ Fabriqué pour:

A&D Engineering, Inc.

4622 Runway Boulevard

Ann Arbor, MI 48108 USA

andmedical.com

Phone / telephone: 1-888-726-9966

Distributed in Canada by/ Distribué au Canada par:

Auto Control Médical An A&D Company/ Une Compagnie A&D

6695 Millcreek Drive, Unit 6 Mississauga, Ontario L5N 5R8 Canada

andmedical.com

Toll-free / Sans frais 1-800-461-0991