

# VOICE GUIDED Blood Pressure Monitor Premium Upper Arm (UA-1040TBLE)

ENG

1 Insert batteries or plug in AC adapter.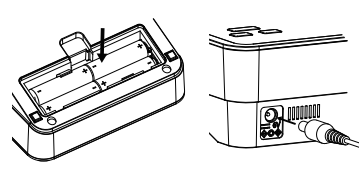

2 Insert the air connector plug into the air socket.

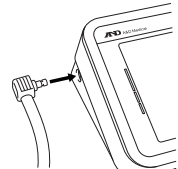
3 Turn on *Bluetooth*® settings on your mobile device (smartphone or tablet).

4 Download the free **A&D Heart Track** app from the App Store or Google play.


5 Getting started with A&D Heart Track app.  
Note: the app screens are subject to change based on design improvements.


5a. Set up account. Create a username and password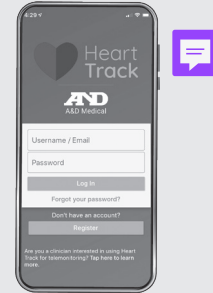


5b. Select Menu

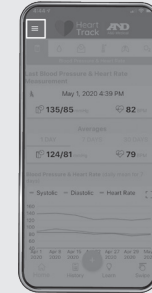
5c. Click Add Bluetooth Devices

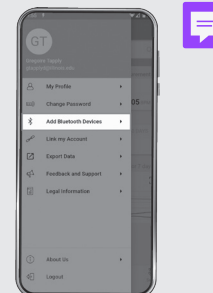5d. Click + and select blood pressure monitor

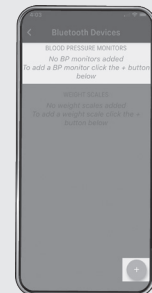
5e. Select UA-1040TBLE 5f. Complete the pairing process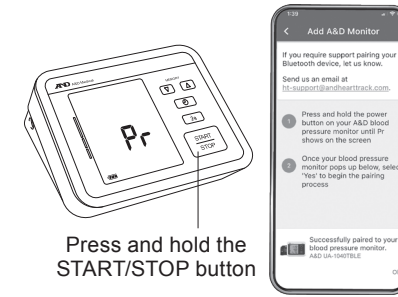


5g. After pairing is completed the monitor turns off and enters standby mode.


5h. Click + at the bottom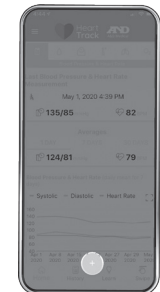

5i. Select Blood Pressure & Heart Rate. Select Auto Bluetooth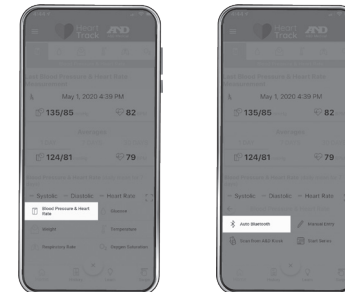

6 Lay your left arm flat on the table with palm up. Place the cuff on your upper arm.

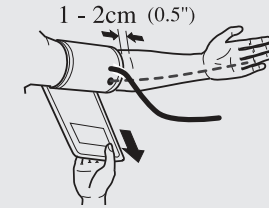7 Fasten the cuff tight enough so you can slide two fingers underneath it.

8 Sit comfortably with your arm resting on a flat surface, elevated to heart level, palm up.


9 Press the START/STOP button.


10 Sit quietly as cuff inflates and deflates.

11 When the measurement is complete, the cuff deflates automatically.

12 Your systolic and diastolic pressure and pulse rate appear on the screen. Measurements are sent to your mobile device when the A&D Heart Track app is open and running. Results are stored in the memory when your mobile device is not available.

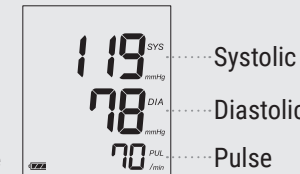## TIPS FOR TAKING BLOOD PRESSURE
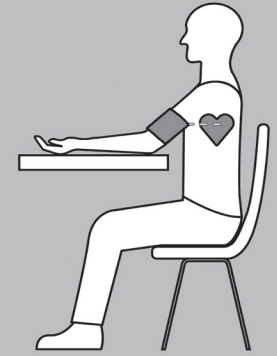
### BEFORE YOUR MEASUREMENT:

- For 30 minutes prior to taking your blood pressure
  - » Do not exercise
  - » Do not drink coffee, caffeinated soda or alcohol
  - » Do not smoke
- Sit quietly for 5-10 minutes
- Ensure you are using the right size cuff

### DURING YOUR MEASUREMENT:

- Do not talk
- Sit with your back straight and supported
- Uncross your legs and place feet flat on floor
- Rest arm on a table so that the cuff is at heart level
- Measure two times a day, in the morning and evening

1-888-726-9966 | andmedical.com

© 2024 A&D Medical. All rights reserved. Specifications subject to change without notice. iPad, iPhone and iPod are trademarks of Apple, Inc. Android and Google play are trademarks of Google, Inc. Specifications subject to change without notice. UA-1040TBLE-QSG

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