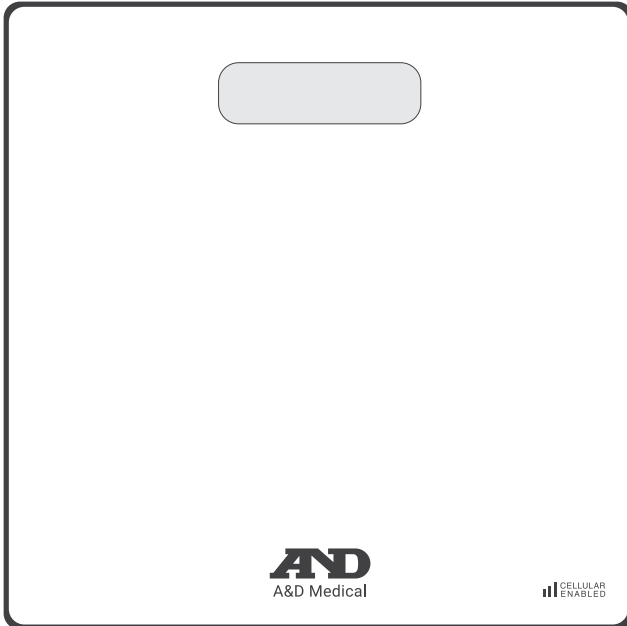
A&D Medical

CELLULAR
PREMIUM WEIGHT SCALE
UC-352CEL

Instruction Manual

Contents

Preface	E-3
General Information	E-3
Safety Information	E-4
Wireless Communication Function.....	E-6
Parts Identification	E-7
Installing (or replacing) the batteries	E-10
Setting the Time.....	E-11
Setting the Units	E-11
Taking a Measurement	E-12
Airplane Mode Setting	E-13
Other Functions	E-13
Troubleshooting	E-14
Maintaining the Scale	E-15
Technical Specifications.....	E-16
Specifications	E-17
Warranty	E-18

Preface

Congratulations on purchasing a state-of-the-art A&D medical connected scale, one of the most advanced health scales available today. Designed for ease of use and accuracy, this device will facilitate your daily weight management regimen. We recommend that you read through this manual carefully before using the device for the first time.

General Information

This scale is intended to be used on a flat, hard surface. If using on a soft surface, such as carpet, it may cause inaccurate measurements.

Safety Information

The following items contain information to prevent the risk of harm or injury to yourself or other people. Please read these items carefully and be sure to use this product correctly.

 Danger	This symbol indicates an immediate danger which could result in death or serious injury.
 Warning	This symbol indicates the possibility of death or serious injury.
 Caution	This symbol indicates the possibility of physical injury or damage to property.

Precautions For Use

Warning

- Do not rely on self-diagnosis when attempting weight loss or therapeutic exercise. Follow the instructions of a physician or specialist.
- Do not step on the edges of the scale or jump on the scale. It may result in a fall, causing injury. The impact may also cause damage to the scale.
- Do not use the scale on an uneven surface. It may damage the scale. Also, the scale cannot measure accurately.
- Do not use the scale on a slippery surface or step on the scale with wet feet. It may result in slipping and injury. Also, the scale cannot measure accurately.

Caution

- Do not place the scale where it is exposed to excess water or humidity such as a bathroom, where it is exposed to direct sunlight, where it is exposed directly to the air outlet of an air conditioning unit, or near fire. It may cause the scale to malfunction.
- This scale should only be used for weighing body weight.
- This scale is precision measuring device. Do not drop the scale, or expose it to vibration or powerful shocks.

Safety Information

Caution

- Do not disassemble, repair or modify the scale. It may cause malfunction or injury.
- Person with disabilities should be assisted by a caregiver when using this scale. Failure to do so may result in a fall, causing injury.
- When installing the batteries, be sure that the + (positive) and - (negative) terminals are correct. Installing incorrectly may cause overheating, battery fluid leaks or rupturing of the batteries, resulting in damage to the scale or injury.
- This scale is designed for personal use.
- Store this scale out of the reach of small children.
- Do not store this scale vertically. If for any reason this scale is stored vertically, be sure not to let the scale fall.
- Do not place any items on this scale during storage.
- If use of this scale is shared with a person who has an infectious disease such as a skin disease on the feet, etc., clean the scale before use. (For details on cleaning the scale, see “Maintaining the Scale (P.13)”).
- To accurately measure your weight, use the scale on a flat, hard floor. Do not use on a carpet or other soft surface.

Wireless Communication Function

This scale uses cellular wireless connectivity to transmit measurement data. At the end of measurement, the measurement data is sent from this scale to the cloud database.

This device complies with Part 15 of FCC rules and contains the FCC ID XMR2021BG770AGL. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operations.

This device complies with Industry Canada licence-exempt RSS standard(s) and contains IC 10224A-2021BG770A. Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

Precautions For Wireless Functions

 Warning
<ul style="list-style-type: none">• Do not use in places where wireless communication is prohibited, such as on airplanes or in hospitals. This scale may have an adverse effect on electronic devices or medical electrical equipment.

Note: A&D cannot accept any responsibility for any losses incurred such as operating malfunctions or loss of data that may occur through the use of this scale.

Parts Identification

Components

- Digital Weight Scale
- 4 AAA Batteries
- Instruction Manual
- Quick Start Guide

Scale

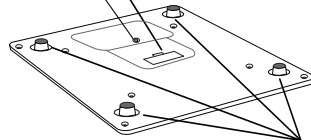
Display

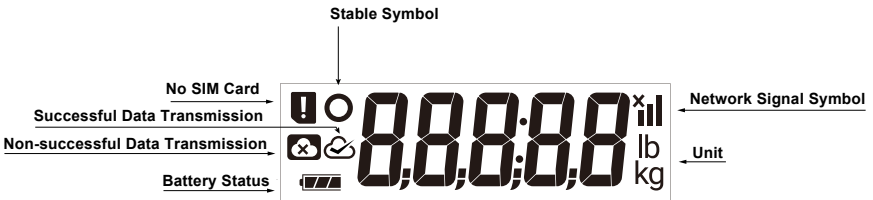
Battery compartment

Set switch

Feet

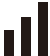
Display

Parts Identification

Display Symbols

Symbol	Description	Recommendation
	Ready/Stable symbol.	This symbol is displayed when the measurement display is stable.
	Batteries are full	_____
 Flashing	Low battery indicator. Batteries are depleted.	If this symbol is displayed, replace all four batteries with new ones.
		
	When blinking, measurement(s) are in process of transmitting measurements to cloud.	_____
	Measurement(s) have successfully transmitted to the cloud database.	_____
	Measurement(s) have not transmitted to the cloud database.	Move closer to a window or an area with a stronger network connection. Wait and take another measurement. Check to see if device is in airplane mode and turn off. If it appears repeatedly, please contact customer service.
	When blinking, network is connecting.	_____
	Network is connected.	_____
	Network is not connected.	Move closer to a window or an area with a stronger network connection. Wait and take another measurement. Check to see if device is in airplane mode and turn off. If it appears repeatedly, please contact customer service.




Parts Identification

Display Symbols

Symbol	Description	Recommendation
	SIM not available/not supported.	<p>Move closer to a window or an area with a stronger network connection. Wait and take another measurement.</p> <p>Check to see if device is in airplane mode and turn off.</p> <p>If it appears repeatedly, please contact customer service.</p>

Installing (or replacing) the batteries

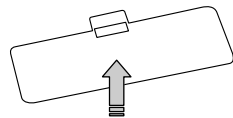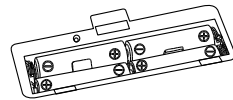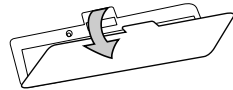1. Remove the battery cover on the back of the device.
2. Insert the 4 AAA batteries with the positive (+) and negative (-) terminals properly aligned with the corresponding symbols in the battery compartment.
3. Close the battery cover.

Caution

- Install each battery with the positive (+) and negative (-) terminals properly aligned with the corresponding symbols in the battery compartment.
- Improper installation of the batteries may cause battery leakage.
- Replace all four batteries with new ones, when the  symbol is displayed. Do not mix new and used batteries. It may cause damage to the scale or shorten the battery life.
- Be sure to use AAA size alkaline batteries. Do not use any other type of battery.
- Never use rechargeable batteries with this scale.
- The battery life is dependant upon the environmental temperature, battery type and usage.
- The batteries provided with the device are for testing the scale performance and may have a limited life.
- Remove the batteries from the battery compartment, when the scale is not to be used for a long time. Storing the scale with the batteries installed may result in battery fluid leaking from the batteries, causing damage to the scale. The supplied batteries are for checking operation of the scale and may have a short lifespan.

Setting the Time

1. Insert battery.
2. "--:--" will blink as the scale establishes the connection and obtains the clock from the network.
3. Clock is displayed.

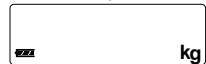
Note: If the scale does not establish a connection, then the network error icon will be displayed.

Setting the Units

1. Confirm that the scale is turned off, and press the set switch.
2. Press the set switch to select the unit that you want to set.
 - The unit changes each time the set switch is pressed.
 - Wait for 3 seconds after setting the unit.
3. Confirm that "End" is displayed.
 - The settings are saved.
 - After three seconds of non-operation, the scale confirms the selected unit with two beeps, displays "End " and automatically turns off.
4. If you wish to delete any readings stored in memory press the set switch. Changing the units format will delete any stored readings on the scale.

♪ Beep

♪ Beep beep

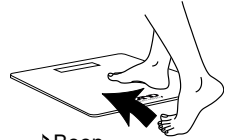
Taking a Measurement

Step On


1. Confirm that the scale is turned off, and step on the scale gently.

- Measurement starts when you step on the scale.
- A progress bar is displayed during measurement. Stand up straight and remain as still as possible during measurement.

2. Step off the scale when “StEP” and “oFF” are alternately displayed with three beeps.

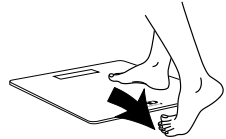- If you do not step off, the scale beeps every two seconds.
- All display segments are indicated when you step off the scale.
- Do not touch the scale while all display segments are displayed. It may cause misreading.

3. Confirm the measurement results.

- The scale turns off automatically 60 seconds after the measurement results are displayed.
- Data is automatically sent to the cloud database.
- If the scale is not able to connect to the cloud, then the data will be stored in the scale memory. (See “Transmitting Temporarily Stored Data”.)
- Data stored in the scale memory cannot be viewed on this scale.
- You can also shut the power off by pressing the set switch.
- The most recent readings will be shown on the screen.

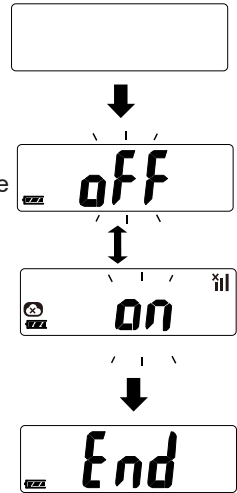
Airplane Mode Setting

Airplane mode setting will disable the network connection and conserve battery for users that are no longer enrolled in a RPM program. **Do not turn airplane mode on if you are still enrolled in a RPM program.**

1. Press and HOLD the set switch for 5 seconds.
2. “oFF” will be displayed and flash. Press set switch to toggle between “oFF” and “on.”
3. After confirming setting, wait 3 seconds and “End” will be displayed. The setting is saved.

Note: When airplane mode is “on,” the network is disabled. The  and  symbols will be always be displayed. Automatic clock will not be set and measurements will not be transmitted to provider. Do not turn airplane mode “on” if you are still enrolled in a RPM program.

Other Functions

Automatic Power Off Function

The scale has an “Automatic power-off” function. After the measurement results are displayed for 60 seconds, the scale is automatically turned off.

Transmitting Temporarily Stored Data

If measurement data cannot be transmitted to the cloud, this scale can temporarily store data of a maximum of 90 measurements in the built-in memory. When the scale is able to transmit to the cloud, all of the data is transmitted. If the amount of data temporarily stored exceeds 90 measurements, the oldest data is deleted and replaced with new data.

- Stored data cannot be viewed on this scale.
- The amount of data that can be temporarily stored may vary with the application.

Troubleshooting

Error Prompt

Error	Corrective Action
	Remove and reinstall the batteries. Or replace all four batteries with new AAA size alkaline batteries.
Nothing is displayed when the setting switch is pressed.	
The measurement results are greatly different from the expected values.	Place the scale on a solid level surface.
	Remove any foreign objects that might be touching the bottom of the scale.
E	Exceeds the maximum capacity (450lb/200kg).
E-0	Exceeds the maximum capacity (450lb/200kg).
Err	Exceeds the maximum capacity (450lb/200kg).
E-10	Communication error. Remove and reinstall the batteries.
E-11	

Maintaining the Scale

- If the scale becomes dirty, soak a cloth in water or mild detergent, wring the cloth out thoroughly and then wipe the scale clean.
- After wiping away the dirt with the damp cloth, dry the scale using a soft, dry cloth.
- Never use alcohol, benzene, thinner or other harsh chemicals to clean the scale.
- Remove the batteries from the battery compartment, when the scale is not going to be used for a long time.

Technical Specifications

Product Name	UC-352CEL
Maximum Capacity	450 lb / 200 kg
Minimum Resolution	0 to 220 lb with an increment of 0.2 lb 220 to 450lb with an increment of 0.5 lb 0 to 100 kg with an increment of 0.1 kg 100 to 200 kg with an increment of 0.2 kg
Wireless Communication	LTE™-M (LTE™-Cat.M1) NB-IoT (LTE™Cat.NB1)
Setting Units	Units: lb or kg
Power Source	4 AAA size alkaline batteries
Dimensions (WxDxH)	11.8 x 11.8 x 1.2" (300 x 300 x 31 mm)
Weight	4.7 lb (2.1kg) including batteries
Accessories	1 Instruction Manual 4 "AAA" Batteries 1 Quick Start Guide

Specifications

FCC Caution

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This transmitter must not be co-located or operated in conjunction with any other antenna or transmitter. This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment and meets the FCC radio frequency (RF) Exposure Guidelines as this equipment has very low levels of RF energy. But it is desirable that it should be installed and operated keeping the radiator at least 20cm or more away from person's body.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help

Warranty

LIMITED WARRANTY

A&D Medical ("A&D")

For purchasers within the US and Canada only:

Product	Warranty Term
UC-352CEL	2 Years

Limited Warranty:

A&D Medical ("A&D") warrants to the first purchaser ("You") that the A&D product You purchased (the "Product") will be free from defects in material, workmanship and design for the applicable Warranty Term stated above from the date You purchased the Product under normal use. This Limited Warranty is personal to You and is not transferable. If the Product is defective, then You return the Product to A&D in accordance with the procedure set forth below. A&D's warranty obligation is limited to the repair or replacement, at A&D's option, of the defective Product that has been returned by You within the warranty period. Such repair or replacement will be at no charge to You. The repaired or replacement Product is warranted here-under for the longer of the remainder of the original warranty period or 90 days from the date of shipment of the repaired or replacement Product. To obtain a warranty service, please contact us in **US at 1-888-726-9966 or in Canada at 1-800-461-0991** for return address, shipping and handling fee, and other instructions for processing warranty. Please ensure you have satisfactory proof of the date of Your purchase and a description of the defect. Returns will not be accepted unless a Return Material Authorization (RMA) Number has been issued from A&D Customer Service Representative.

This Limited Warranty does not cover, and A&D will not be liable for (i) any shipment damage, (ii) any damage or defect due to misuse, abuse, failure to use reasonable care, failure to follow written instructions enclosed with the Product, accident, subjecting the Product to any voltage other than the specified voltage, improper environmental conditions, or modification, alteration or repair by anyone other than A&D or persons authorized by A&D, or (iii) expendable or consumable components.

Warranty

THIS LIMITED WARRANTY IS THE ONLY WARRANTY PROVIDED BY A&D; THERE ARE NO OTHER EXPRESS WARRANTIES. If A&D cannot reasonably repair or replace the Product, A&D will refund the amount You paid for the Product (not including taxes), less a reasonable charge for usage. To receive a refund you must have returned the Product and all associated materials to A&D. The above remedy of repair, replacement or refund is your only and exclusive remedy. IN NO EVENT SHALL A&D BE LIABLE FOR ANY DAMAGES, INCLUDING, WITHOUT LIMITATION, LOST PROFITS, LOST INFORMATION OR REPLACEMENT COSTS, ARISING OUT OF YOUR USE OF OR INABILITY TO USE THE PRODUCT, INCLUDING, WITHOUT LIMITATION, ANY SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES, EVEN IF A&D HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. Some states do not allow the exclusion of incidental or consequential damages, so that the above exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that may vary from state to state.

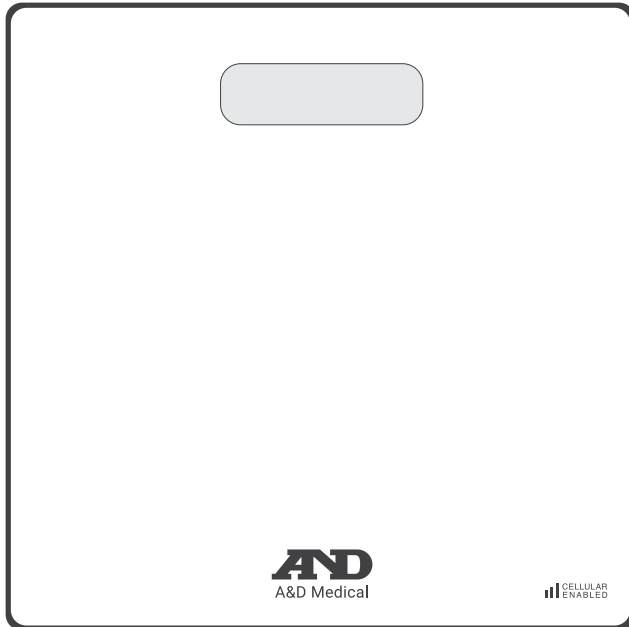
No distributor, dealer or other party is authorized to make any warranty on behalf of A&D or to modify this warranty, or to assume for A&D any liability with respect to its products.

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A&D Company, Limited
A&D Engineering, Inc.