


WIRELESS
BODY COMPOSITION SCALE
WITH A&D MEDICAL HEART TRACK
UC-450BLE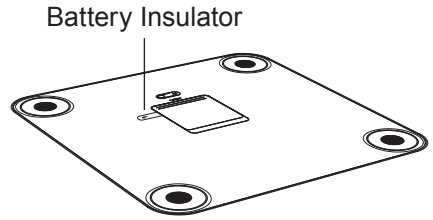

1. The scale comes with batteries pre-installed, please remove the battery insulator for first use.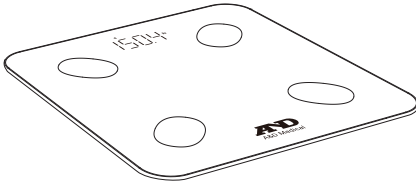

2. To change the units, push and release the set switch. Default is lb.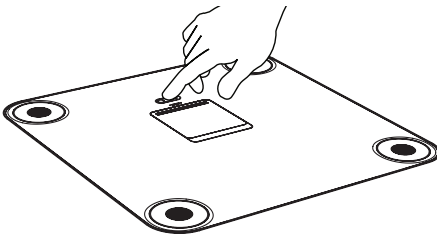

3. Turn on *Bluetooth*[®] settings on your mobile device (smartphone or tablet).
4. Download the free **A&D Heart Track** app from the App Store or Google Play.


GET THE APP

5. Getting started with **A&D Heart Track** app.

Note: the app screens are subject to change based on design improvements.

5a. Set up account. Create a username and password.

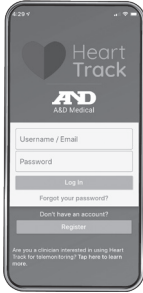5b. Select Weight Icon.

5c. Click Add Bluetooth®
Weight Scale.

5d. Select UC-450BLE.

5e. Follow onscreen instructions to complete pairing.

Note: tap on scale to wake it up prior to pairing - the LED will light up.


6. Start measuring.

Note: If you take measurements prior to pairing, the date and time might be inaccurate for measurements taken prior to pairing.

6a. Open the A&D Medical Heart Track app.

6b. Place the scale on a flat and stable surface.

6c. Click **+Measure** button on app.

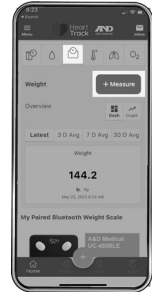
6d. Step on the platform barefooted.

6e. Stand still and keep full contact with the electrodes until the LED stops blinking digits.

Note: Do not step off the scale until the display stops flashing. When the display is no longer flashing, the measurement is complete.

6f. Wait for the scale to transmit measurements to the app.

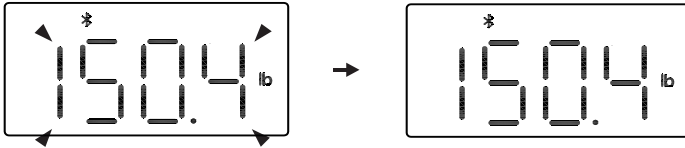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©2024 A&D Medical. Tous droits réservés. Les spécifications peuvent être modifiées sans préavis. iPad, iPhone et iPod sont des marques de commerce d'Apple, Inc. Android et Google Play sont des marques de commerce de Google, Inc. Les spécifications peuvent être modifiées sans préavis.