



Cleaning and Disinfection Recommendations for TM-2657P, TM-2440, TM-2441, UM-211, UM-212BLE and Cuffs

Note: If the cuff is damaged, worn or too dirty to clean, please contact customer support to purchase a new cuff or a cuff replacement sleeve.

Cleaning the Main Body and Cuff

To clean the main body and the cuff, use a soft cloth soaked with a neutral detergent solution and warm water, then gently wipe down. Dry with a soft cloth.

To Disinfect the Main Body

To disinfect the main body, use a soft cloth soaked with an antiseptic solution. The antiseptic solution should be a water solution with the provided dilution ratio of one disinfectant, as shown in the below table. Use a different soft cloth, soaked with lukewarm water to gently wipe down the main body. Dry with soft cloth.

Component Name	Dilution %
Ethanol	76.9-81.4%
Isopropanol	70%
Aqueous Solution of the Benzalkonium	0.10%
Chlorhexidine Gluconate Solution -	0.5%
Sodium Hypochlorite	0.05%
A surfactant (detergent for the kitchen)	

To Disinfect the Cuffs

To disinfect the cuff, use a soft cloth soaked with an antiseptic solution. The antiseptic solution should be a water solution with the provided dilution ratio of one disinfectant, as shown in the below table. Then, use a different soft cloth, soaked with lukewarm water to gently wipe down the main body. Dry with soft cloth.

Component Name	Dilution %
Ethanol	76.9-81.4%
Isopropanol	70%
Aqueous Solution of the Chlorhexidine gluconate	0.5%
Aqueous Solution of the Benzalkonium chloride	0.1%